

Some Other Popular Toys

Pouring Cup #20

Ring Around Bells #23

Dome Alone #69

Dome Alone with Switch #71

Drum #80

Musical Rolling Pin #160

Musical Light Box #200

Stacking Blocks #359

Stacking Ring Cone #360

Musical Top #400

Vibrating Light with Music #459

Glitter Roll #696

Motorized Glitter Roll #697

Butterfly Light and Music Puzzle #922

Traffic Light #1800

String Thing with Lights

Catalog #490-L

User's Guide



Enabling Devices

50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com



Enabling Devices

50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

String Thing with Lights

Operation:

1. The String Thing requires 4AA batteries (Not included). To install new batteries, first turn the String Thing over. Next simply unscrew the Philips screw on the case and, lift the battery off. Insert new batteries being sure to observe proper (+) & (-) polarity. Close cover and replace screw.
2. Set the String thing in either the Black-light or String position.
3. Plug your external capability switch into the female jack located on the side of the String Thing. When your switch is pressed, the String Thing will activate. Once you release your switch, the String Thing will turn off. The String Thing can be used in or out of the display base.

Please Note:

Due to switch adaptation the String thing will only work with an external switch. It will no longer work by the use of the built-in ON/OFF switch.

During operation if the string gets tangled or is stopped by the user pulling on it, it will stop as a safety feature to prevent burning out the motor.

Troubleshooting:

Problem: The String Thing does not turn on when switch is depressed.

Action #1: Make sure that you press all connections firmly together. Any gaps may interrupt electrical connection. Also the String & Blacklight/String/Off switch needs to be in the String or Blacklight & String position to work with a switch.

Action #2: Make sure batteries are fresh.

Action #3: Try a different switch with the String Thing to rule it out as the source of the problem.

Care of Unit:

The String Thing can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit in any fluids, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697
customer_support@enablingdevices.com