

---

---

## **Crazy Chicken- #3064**

---

---

## **USER'S GUIDE**

---

### **For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-TEC-TOYS or 1-800-832-8697  
[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)

---



**Enabling Devices**

50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

*Dances the chicken dance!*

Hit your switch and the crazy chicken flaps his wings and waddles around while doing the chicken dance in sync with the music. Also works when grasping his neck or pinching his wing. Size: 12 1/2"H. Requires 4 AA Batteries.

**Operation:**

1. Crazy Chicken requires 4 AA batteries (Not included) to operate. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. The battery compartment is located in Crazy Chicken's back. Open the Velcro flap then, using a Phillips head screwdriver open and remove the batter cover. Next install 4 AA batteries paying close attention to proper battery polarity.
3. Replace the battery compartment cover and secure it with the screw. Turn the AUTO/OFF/ON switch to ON, which is located on the bottom of Crazy Chicken.
4. Connect Crazy Chicken to any external capability switch (not included) via the jack located at the end of his cord. Crazy Chicken is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure you have a tight connection. There should be no gaps.
5. Actuate your external capability switch and Crazy Chicken will come to life! This Crazy Chicken flaps and waddles around while doing the Chicken Dance, clucking and flapping its wings in sync with the music. Lift him by the neck while he dances and he squawks and clucks like mad, flapping his wings and feet wildly as if he's gagging and choking. But not to worry -- just set him back down and he continues to dance!

6. Press your switch again and he will start dancing again. You can also play with Crazy Chicken without an external capability switch. Simply press his left wing, and he will do his dance.
7. When you are finished using Crazy Chicken turn his ON/OFF switch to the OFF position.

**Troubleshooting:**

**Problem:** Crazy Chicken does not function correctly.

**Action #1:** Replace the batteries paying close attention to proper battery polarity.

**Action #2:** Make sure that the connection between the external capability switch and the jack is TIGHT. There should be NO gaps.

**Action #3:** Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

**Care of Unit:**

If you will not be using Crazy Chicken for an extended amount of time, we recommend removing the batteries.

**Do not submerge** the unit in water, as it will damage the electrical components. Crazy Chicken is surface washable only.