

Troubleshooting:

Problem: The Frog Puzzle Communicator does not function correctly.

Action #1: Check and or replace the batteries paying close attention to proper battery polarity.

Action #2: Make sure that all the shapes are being pushed in all the way to play back the messages.

Action # 3: Make sure there is nothing blocking or inside the shape location areas.

Care of Unit:

When finished using the Frog Puzzle Communicator turn the ON/OFF/VOLUME knob to the OFF position.

If you will not be using the Frog Puzzle Communicator for an extended amount of time, we recommend removing the batteries.

The Frog Puzzle Communicator can be wiped clean with any mild household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

**Do not use abrasive cleaners, as they will scratch the surface of the unit.
Do not submerge the unit, as it will damage the electrical components.**

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697
customer_support@enablingdevices.com

Frog Puzzle Communicator #476

USER'S GUIDE


Enabling Devices

50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Unique frog sounds!

Our five piece take or place Frog Puzzle is so much fun. The first two levels already have pre-recorded realistic frog sounds from all around the world. The remaining eight levels are for your own recordings. We added knobs for easy placement. Five seconds of record time per message. Size: 11 1/2"W x 4"H x 8"D. Requires 4 AA Batteries.

Operation:

1. Frog Puzzle Communicator requires 4 AA batteries. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.

1. Turn the Frog Puzzle Communicator over to reveal the battery compartment. Using a Phillips head screwdriver open the compartment and install 4 AA batteries paying close attention to proper battery polarity. Replace the battery compartment cover and secure it with the screw.

2. Turn the Frog Puzzle Communicator ON via the ON/OFF VOLUME knob located on the bottom of the unit. Set the volume to an audible level. Please Note: When the unit is first turned ON it will beep once to indicate it is on Level 1 this is normal. If you press the Level changer button on the back of the unit it will beep twice to indicate you have now advanced to Level 2. The first two levels are pre-recorded with 10 unique frog sounds from all over the world. Every time you place or take out a piece of the puzzle it will play one of the wonderful frog sounds. It also has green location pins in the frog to hold each piece in place. The next 8 levels are for the user to record his or her own messages. Each message has a maximum recording time of 5 seconds. PLEASE NOTE: The first two level recordings cannot be changed or erased.

3. The communicator will always default to Level 1 when first turned on. The unit will beep once; press the record button the unit will beep twice. Press the record button once more to get you to Level 3. Next record each level announcement do the following steps press the record button located on the bottom of the unit, then while holding the Record button depress the Level Changer button and speak your message into the Microphone which is the small black fuzzy circle. Release both when done. Your mouth should be no more than 6 to 8 inches away from the microphone. Please Note: After Recording the Level announcement, you will not hear what you just Recorded when you press the Level Changer button again. You will have to cycle through all the Levels until you get back to Level 3. The Level 3 message is now recorded. To record Level 4, press the Level Changer button once you may hear a previous recorded message now press and hold the Record button, then as before depress the Level Changer button and speak your message. Follow the same steps for recording the remaining Level announcements.

4. Helpful Recording Tips: If you are in doubt you recorded the Level announcements properly, turn the unit OFF then turn it back ON and press the Level changer button twice, the message announcement you first hear should be what you recorded for Level 3. If not start over again as in operation No.3.

5. To record your messages, set the unit to the proper level (e.g. level 3 will be Frog Colors). Next depress and hold the **RECORD** button, then depress one of the color coded message buttons and speak into the microphone. (The Puzzle pieces have corresponding colored dots on the back of them; record your messages to correspond). Release both buttons once recording is over. Test playback of your message by pressing and releasing the same colored button used to record. (The unit will retain recorded information indefinitely, even if the batteries are removed). Previously stored information will be erased only when a new recording is performed.) Each message has a maximum recording time of 5 seconds.

7. Each time you Place or Take (depending on what setting you have the unit set to) a puzzle piece from one of the location areas, the message recorded for that location will play. Make sure when inserting a puzzle piece that it is pushed in all the way.

Important Note: Each puzzle and puzzle pieces are not interchangeable with other sets of like or similar puzzles.