

---

---

## **Balls Go Round #1813**

---

---

### **USER'S GUIDE**

---

**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-TEC-TOYS or 1-800-832-8697  
[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)

---

  
**Enabling Devices**

50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

*Fantastic visuals!*

Balls-go-round is a great toy for teaching cause and effect and to increase visual attention. It has 4 New Wave upbeat tunes, multi-colored blinking lights, adjustable volume, and a tactile pad. It also features adjustable rotation speed, so you can control how fast it turns in a circular pattern. Has a jack input for switch use. Size: 18"H x 16"W x 10 1/2"D. Requires 6 AA and 2 C Batteries.

1. The Balls Go Round requires 6 AA batteries and 2 C batteries (not included). Alkaline batteries must only be used (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. The battery compartments are located on top back of the unit. Turn the unit around and remove the Velcro attached cover to reveal the 3 black battery compartments. Please Note: The C batteries have no additional cover. Carefully remove the small screw from each of the battery compartment covers using a Phillips head screwdriver, and then slide the battery compartment covers off. Install AA size batteries into each holder. When installing new batteries, always be careful to observe battery polarity. Replace battery covers and screws. Never mix old and new batteries together or different brands or types together.
2. Turn the ON/OFF/VOLUME knob clockwise, you will hear an audible click. Adjust to knob for desired volume level.
3. There is also a SPEED control knob which controls how fast the balls go round, turning the knob clockwise increases the speed, counter clockwise decreases the speed.
4. Simply press on the round yellow switch plate and you will be rewarded with flashing multi colored lights, music and spinning balls. The Lights, Music and balls will run for approximately 10 seconds with each switch activation. Each time you activate your switch the music will change. Please Note: cannot be used with a switch Latch/Timer switch in the Latch/Timer mode.
5. To use the Balls Go Round with a switch, plug, any Capability switch into the switch jack located on the side of the Balls Go Round. Make sure your connections are flush with no gaps.
6. Activate your switch and you will be rewarded as noted is step 3.

**Wall Mounting:**

The Balls Go Round comes with wall mounting hardware. To mount to a wall you must first remove the unit from its Velcro attached base, next turn the unit over there are 4 large Phillips head screws on the back panel. Remove the 4 screws and then place the wall mounting plate so that the 4 inner screw holes on the plate line up with the screw holes from the removed screws. Replace the screws to secure the mounting plate to the back of the unit. Next attach the two threaded posts to the screw holes at the bottom of each side of the unit. These posts are needed to keep the unit level when mounted to the wall. Use the include wall mounting screws and wall anchors to attach the unit to the wall.

**Troubleshooting:**

**Problem:** Balls Go Round does not spin or light up.

**Action #1:** Check the Batteries for strength and for proper battery polarity. Replace if weak or dead.

**Problem:** Balls Go Round does not play music or light up.

**Action:** Check the Batteries for strength and for proper battery polarity. Replace if weak or dead.

**Problem:** Balls Go Round does not work at all.

**Action:** Check all Batteries for strength and for proper battery polarity. Replace if weak or dead.

**Problem:** Balls Go Round does not activate by an external switch.

**Action # 1:** Make sure all connections are tight.

**Action # 2:** Check your switch with another device you know is working.

**Action # 3:** If an adapter is being used, it must be Mono, stereo adapters will not work.

**Care of the Unit:**

Balls Go Round can be wiped clean with any household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

**Do not use** abrasive cleaners, as they will scratch the surface of the unit.  
**Do not submerge** the unit, as it will damage the electrical components.

Rev 3/2/09