

Watch the swirling shapes, as you play your message!

Pressing the dome rewards the user with vibration and your own pre-recorded message, (up to 20 seconds). It is easy to record and re-record any songs or words by pressing a hidden switch.

Size: 8"W x 4½"H • Requires 6AA Batteries (Not included)

OPERATION:

1. Carefully turn the unit over to reveal the battery compartments. Open the battery compartment covers by removing the small Phillips head screws, and install six "AA" size batteries. Replace covers.
2. The **On/Off/Volume** knob and spinner **On/Off** switch are located on the back of the unit. The pushbutton **RECORD** switch is located on the bottom panel of the unit **Please Note:** Make sure the Spinner switch is turned to the **Off** position before recording. Failing to do so will cause the Spinners background noise to be recorded. To record, depress and hold the **RECORD** button, then depress the Dome and speak 6 to 8 inches from the microphone, located next to the **On/Off/Volume** knob. Release both **RECORD** button and Dome once the recording is over. Playback your message by pressing and releasing the Dome.
3. The unit will retain recorded information indefinitely, even if the battery is disconnected. Previously stored information will be erased only when new recording is performed.
4. Playback volume can be adjusted using the control provided on the back panel of the unit.
5. The spinner inside the clear dome can be turned **On** or **Off** during activation of the Dome via the On/Off Toggle switch on the side of the unit.
6. Attach the cord (included) from the Saucer Dome via the jack located on the back of the unit to a toy/device. When the Dome is pressed the message plays and toy/device is activated simultaneously.

IMPORTANT NOTES:

1. The memory of this unit is capable of holding up to 20 seconds of speech information. If the message length exceeds available memory amount (20 seconds), only the first 20 seconds will be recorded.

Troubleshooting:

1. If the unit fails to operate, or operates incorrectly, the following steps should be performed.
 - Disconnect the power by removing one of the batteries for 20 seconds to reset the unit. If the unit still does not work after the battery is reconnected, replace all the batteries.
 - Make sure there are no obstacles between the plate and the housing of the unit.
 - When connecting a toy/device make sure there are no gaps between any of the connections. All should be flush.
 - Try your toy/device with a different switch to rule out the Saucer Dome as the source of the problem.

If you have any problems or questions about this unit or any of our products, please call our **Technical Assistance Department** Monday through Friday, 9:00 a.m. to 5:00 p.m. (EST):

1-800-TEC-TOYS
(1-800-832-8697)

Some Other Popular Toys/Switches

Somatosensory Bead Chain with Music Switch #357

Deluxe Razzle Dazzle Bead Chain Curtain #361

Glitter Roll Music Box/Switch #696

Oval Texture Music Box/Switch #712

Oval Texture Multi-Sensory Center/Switch #725

Sensiball Say-It-Play-It #666

Sensiball Say-It-Play-It (48 Messages) #667

Gooshy Say-It-Play-It #671

Gooshy Say-It-Play-It (48 Messages) #672

Koosh™ Switch #883

1-, 2- and 3-Shape Switch/Music Box #900, 910, 920

1-, 2- and 3-Shape Switch #930, 940, 950

Interchangeable Music Puzzle/Switch #921

Lighted Signal Switch #956

Bitty Bubble #5100

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697

Saucer Dome Say-It-Play-It

Catalog # 9158

User's Guide

**Enabling Devices**

50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com