

Paw Switch - #5586

USER'S GUIDE

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697
customer_support@enablingdevices.com


Enabling Devices

50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Encourages Bi-Lateral hand usage!

We have developed a very unique three way output switch. With the first two outputs you can activate the right or left 3" paw pads and it will activate one or two toys. In the third output you must activate both switches at the same time for the toy to work. Size: 11 ½" W 2" H x 5" D

Operation:

1. Connect the Paw switch to any adapted toys/devices, via the jacks, located on the base of the unit. Plug one end of the supplied 1/8" to 1/8" cord into the jack on the base of the Paw Switch and the other end into your toy/device. Make sure all connections are tight, there should be no gaps.
2. Pressing the left Paw will activate one toy/device for as long as the Paw switch is depressed. Once the Paw switch is released, the connected toy/device will stop.
3. Pressing the right Paw will activate one toy/device for as long as the Paw switch is depressed. Once the Paw switch is released, the connected toy/device will stop.
4. Pressing both the left and right Paw will activate a third toy/device for as long as both Paw switches are depressed. Once the Paw switch is released, the connected toy/device will stop.

Troubleshooting:

Problem: The Paw Switch does not work.

Action #1: Make sure that the connection between the Paw Switch and your toys/devices are tight. There should be *no* gaps. This is a common error and an easy fix.

Action #2: Check to make sure that no debris or small objects are blocking or obstruction any of the Paw switch plates.

Action #3: Try a different switch with your toys/devices to rule out the toys/devices as the source of the problem.

Action #4: Try a different adapter (if applicable) to rule this out as the source of the problem.

Care of the Unit:

The Paw Switch can be wiped clean with any mild household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.