

Our popular 6-Level Communicator and 6-Level Communicator w/Jacks now have an easy level changer feature. One switch is now dedicated to changing levels. Therapist records the name of each level and the user simply pushes the icon to select desired level. Its contents are spoken to the user. Record time is 6 seconds per message.

Size: 12 1/2"L x 8"W x 3"H. Requires 4 AA Batteries.

OPERATION:

1. Carefully turn over unit to reveal battery compartment. Observing polarity, install four AA batteries into the holder.
2. Turn the unit on. The **LEVEL** will always be Level one when the unit is first turned on. To record each level announcement do the following steps press and hold the record button first, then press the Level Changer and speak your message, release both when done. Each time you press the **LEVEL** button it will advance to the next level, and then you can press and hold the **REC** button then press the level button to record that levels announcement.
3. To record your messages, depress and hold the **RECORD** button, then depress one of the 7 squares and speak into the microphone. Release both buttons once recording is over. Playback your message by pressing and releasing the same square used to record, or use an external switch plugged into the corresponding jack. **Please Note: if using external switches to playback messages, eight separate switches will be needed to allow access to all messages.** Each of the remaining buttons can be programmed in the same manner. (The unit will retain recorded information indefinitely, even if the battery is disconnected. Previously stored information will be erased only when new recording is performed.)
4. To move to the next level press the bottom right hand square marked level changer. This will set the unit to level two. Record your level message by pressing only the record button and speaking your message.
5. Record remaining level announcements and messages as in steps 3 and 4. Each time the level changer square is pressed it will advance to the next level and announce your message.
6. Draw or print corresponding icons or words on a sheet of paper measuring 5 1/2" x 10" so they will appear through the 8 cutouts on the housing when inserted.
7. Carefully insert the sheet between the two top cover plates. Slide the page down until your icons or words appear through the 8 square windows.

9. Playback volume can be adjusted using the control provided on the back panel of the unit, next to the **RECORD** button.

10. Turn the unit off by rotating the **VOLUME** control knob located on the back of the unit counterclockwise until you hear an audible "click".

IMPORTANT NOTES:

When changing a Level announcement, you must hear the message first that you want to change before pressing the **RECORD** button and then the Level Changer button.

Additional paper inserts may be cut from regular paper to the recommended size of 5 1/2" x 10" to fit properly into the housing.

Switch input:

There are four external switch input jacks, which allows the user to play back the four corresponding messages via their external switches. PLEASE NOTE: the jacks are not for activating external toys or devices adapted for switch use.

The 6 Level Communicator will shut off automatically after playback to preserve battery life. However, the device still drains a very small current in this "sleep" mode. Therefore, if the unit is not going to be used for a prolonged amount of time (more than a few days), it is recommended to turn **OFF** the unit by rotating the **VOLUME** control knob counterclockwise until you hear an audible "click".

If the unit fails to operate, or operates incorrectly, the following steps should be performed:

- Disconnect the power by turning **OFF** the unit for 20 seconds to reset the unit. If the unit still does not work correctly after the power is reconnected, replace the batteries observing proper polarity.
- Make sure there are no obstacles between the square buttons and the housing unit.

If you have any problems or questions about this unit or any of our products, please call our Technical Assistance Department at:

(1-800-832-8697)

customer_support@enablingdevices.com

Other Communication Devices

Communication Builder #7071

Communication Builder #7071-B (Black)

7 Level Blue Communication Builder #7075

7 Level Black Communication Builder #7075-B

1 Level-32 Message Communicator #3201

2 Level-32 Message Communicator #3202

3 Level-32 Message Communicator #3203

6 Level-32 Message Communicator #3206

Auditory Queuing Communicator 16 level #4399

2 Level Communicator with Overlay Option #2302

2 Level Communicator with 1/8" Jacks And Overlay Option
#2303-8

6-Level Communicator with Overlay #2391

6-Level Communicator with 1/8" Jacks and Overlay Option
#2392-8

Advanced 6-Level Communicator with Jacks

Cat. No. 2392A

User's Guide


Enabling Devices

50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com