
Baby Elephant #958

USER'S GUIDE



Enabling Devices

50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697
customer_support@enablingdevices.com

Trumpet!

Baby Elephant will walk, move his ears and make elephant sounds when you use your capability switch. He is a great animal friend to own. Size: 12"L x 7"W x 8"H. Requires 2 C Batteries.

Operation:

1. Baby Elephant 2 C Batteries to operate. Alkaline batteries must be used (e.g. Duracell or Energizer Brands). Do not use rechargeable batteries because they supply a lower voltage and the unit may not perform as well. Never mix old and new batteries together or different brands or types together.
2. Turn Baby Elephant upside down to reveal the battery compartment. Open battery compartment and install 2 C batteries paying close attention to proper battery polarity. Close the battery compartment.
3. Connect Baby Elephant to any external capability switch (not included) via the jack located at the end of Baby Elephant's cord. Baby Elephant is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure to have a tight connection. There should be no gaps.
4. Actuate your external capability switch and Baby Elephant will spring to life! He will walk, move his ears and make elephant sounds.
5. Baby Elephant will remain on while the external capability switch continues to be actuated. Once you release the external capability switch, there will be a brief delay, and then Baby Elephant will turn off.

Troubleshooting:

Problem: Baby Elephant does not function correctly.

Action #1: Replace the batteries paying close attention to proper (+) & (-) battery polarity.

Action #2: Make sure that the connection between the external capability switch and the jack is TIGHT. There should be NO gaps.

Action #3: Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

Care of Unit:

- Do not inhibit Baby Elephant's movement, as this may cause irreversible damage.
- Do not lift Baby Elephant up by his head, tail, trunk, legs, or drop Baby Elephant.
- If you will not be using Baby Elephant for an extended amount of time, we recommend removing the batteries.

Do not submerge the unit in water, as it will damage the electrical components. Baby Elephant is surface washable only.