

When desired posture is achieved, this trainer plays soft music. Velcro strap enables easy attachment. This unit requires two A76 watch cell batteries (Included).

Operation:

1. This Posture Trainer requires two A76 watch cell batteries. One has been removed to preserve the batteries. **Please Note:** There is no On/Off switch on this device. When storing this unit remove one battery to prevent any accidental activation, which will drain the batteries.
2. To install the batteries turn the unit over so the Velcro backed side is facing up. Next remove the off center Phillips head screw and lift cover off. Install the batteries with the (+) side facing up. Replace cover and screw.
3. Place elastic headband on individual's head with the Velcro tab facing out. Place Musical Posture Trainer on the headband via the Velcro Tab. Adjust angle for desired activation.

Troubleshooting:

Problem: Musical Posture Trainer does not work.

Action: #1

Check to see if both batteries are installed with (+) side facing up, if so and unit still does not work replaces batteries.

Action: #2

Make sure you have adjusted the Posture Trainer properly.

Care of Unit:

The Musical Posture Trainer can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

For Technical Support:

Call our Technical Service Department
Monday Through Friday, 9 a.m. to 5 P.M. (EST)
1-800-TEC-TOYS or 1-800-832-8697

Musical Posture Trainer

Catalog No. 1120

User's Guide

The logo for Enabling Devices features the company name in a bold, serif font. Above the text is a stylized graphic consisting of a curved line that starts on the left, arches over the word "Enabling", and ends with a small star on the right side.

Enabling Devices

50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com