

---

---

# Fishy Fascination Switch #893

---

---

## USER'S GUIDE

  
**Enabling Devices**

50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

---

### **For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-TEC-TOYS or 1-800-832-8697  
[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)

---

Spin the fish carousel and be rewarded with spectacular multi-colored blinking lights, music and vibration. When spinning the fish you can also watch their mirrored reflection as they go around and around. The toy has a control on the bottom to allow you to turn any of the three rewards on or off. Size: 7"L x 5.5"W x 6"H. Requires 4 AA Batteries.

### Operation:

1. The Fishy Fascination Switch requires four AA batteries (Not included). The battery compartments are located under the base of the unit. Carefully turn the unit over, and then remove the Philips head screws from the battery covers. Install new batteries, being careful to observe proper battery polarity. Alkaline batteries must be used only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old, new or different types of batteries together. Replace the battery compartment covers and screws.
2. To turn ON or OFF the desired stimuli (1-Lights; 2-Music; 3-Vibration), push the small white rocker switches to the ON or (Open position). Open position means OFF. Please see Fig 1.

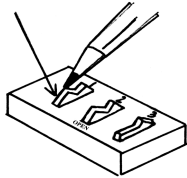


Figure 1.

3. The slightest rotation of Fishy Fascination sets off multi colored lights, music and vibration. Stop rotating or spinning of the Fishy Fascination will cause the Lights, Music and Vibration to stop.
4. To use the Fishy Fascination Switch as a switch plug one end of the supplied jack cord into the Fishy Fascination Switch's female jack and the other end into your adapted toy/device. Rotate or spin the Fishy Fascination and your toy/device will activate as long as rotating or spinning of the Fishy Fascination is maintained.

### Troubleshooting:

**Problem:** The Fishy Fascination Switch has no lights, Music or Vibration.

**Action #1:** Check all batteries for strength and for proper battery polarity. Replace if weak or dead.

**Action # 2:** Make sure nothing is preventing or blocking the Fishy Fascination from rotating.

**Problem:** The Fishy Fascination Switch does not work at all.

**Action:** Check all Batteries for strength and for proper battery polarity. Replace if weak or dead.

**Problem:** Fishy Fascination Switch does not activate a toy or device.

**Action # 1:** Make sure all connections are tight.

**Action # 2:** Check batteries in toy/device, replace if weak or dead.

**Action # 3:** If adapters are being used they must be Mono, stereo adapters will not work.

**Action #4:** Try your toy or device with another switch you know works to rule it out as the source of the problem.

### Care of the Unit:

The Fishy Fascination Switch can be wiped clean with any mild household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

**Do not use** abrasive cleaners, as they will scratch the surface of the unit.

**Do not submerge** the unit, as it will damage the electrical components.