

---

---

## Story Teller - #1022

---

---

### USER'S GUIDE



**Enabling Devices**

385 Warburton Avenue  
Hastings on Hudson, NY 10706  
Tel. 914.478.0960 / Fax 914.478.7030

[www.enablingdevices.com](http://www.enablingdevices.com)

---

**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-TEC-TOYS or 1-800-832-8697  
customer\_support@enablingdevices.com

---

Place any object on the spinning platform with Velcro for object identification. Simply press the oval textured switch and you will be rewarded with lights and your pre-recorded story (total record time 20 seconds). Toy included will vary. Size: 12 ½" L X 9 ¾ "W x 4 ½ "H. Requires 4 AA Batteries.

#### **OPERATION:**

1. Carefully turn the unit over to reveal the battery compartment. Open the battery compartment cover and install four "AA" size batteries (Not included). Replace the cover. **Please Note:** Alkaline batteries are required (E.G. Duracell or Energizer brand) Do not use rechargeable or heavy duty batteries as they will not work with this device. Never mix old and new batteries together.
2. Playback volume can be adjusted using the control provided on the bottom panel of the unit, next to the **RECORD** button.
3. First turn the ON/OFF/Volume knob ON, by turning it clockwise. Set the volume to an auditable level. To record, depress and hold the **RECORD** button, then depress the textured yellow switch plate on the front of the unit and speak into the microphone. Release both button and plate once the recording is over. Playback your message by pressing and releasing the textured yellow plate. When the textured yellow plate is depressed and then released, your recorded message will play accompanied by lights and the spinning platform will spin the object you have placed upon it. Once the messages finishes playing all will stop unit the switch plate is depressed again.
4. The unit will retain recorded information indefinitely, even if the batteries are removed. Previously stored information will be erased only when a new recording is performed.
5. Story Teller comes with a Velcro based toy, which allows you to change the toy or object. To remove the toy, grasp the toy from the spinning platform and gently pull the toy off the platform. To replace the toy press the toy onto the platform, making sure the toy is secured properly before use.

#### **IMPORTANT NOTES:**

1. The memory of this unit is capable of holding up to 20 seconds of speech information. If the message length exceeds available memory amount (20 seconds), only the first 20 seconds will be recorded.
2. The unit will shut off automatically after playback to preserve battery life. However, it still drains a very small current in this "sleep" mode. It is recommended to turn the power off when the device is not in use. This will help extend batter life.

#### **Troubleshooting:**

If the unit fails to operate or operates incorrectly, the following steps should be performed:

**Action:** Make sure there are no obstacles between the textured yellow switch plate and the rotating base.

**Action:** Remove the batteries for 20 seconds to reset the unit. If the unit still does not work correctly after the batteries are re-installed, replace the batteries with new alkaline batteries (E.G Duracell or Energizer Brand).

**Action:** Do not use rechargeable batteries.

#### **Care of Unit:**

The Story Teller can be wiped clean with any mild household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

**Do not submerge** the unit, as it will damage the contents and the electrical components.

**Do not use abrasive cleaners,** as they will scratch the surface of the unit.