

### **Great for providing communication and cognitive support**


This user-friendly teaching tool enhances development of expressive and receptive language skills. Plus, it's great for early speech development and remedial work with older individuals. The Cheap Talk 8 allows you to record and rerecord eight 37.5-second messages as often as you'd like. Each 2" square has an easy-to-remove plastic protective cover to hold words and/or pictures. Activated by pressing one of the eight squares. (Pictures not included).

### **OPERATION:**

1. Carefully turn over unit to reveal battery compartment. Observing polarity, install four AA batteries (Not included) into the holder.
2. The microphone and red **"RECORD"** button are located on the back panel of the unit. To record, depress and hold the **"RECORD"** button, then depress one of the 8 colored squares and speak into the microphone. Release both buttons once recording is over. Playback your message by pressing and releasing the same square used to record. Each of the remaining buttons can be "programmed" in the same manner. (The unit will retain recorded information indefinitely, even if the battery is disconnected. Previously stored information will be erased only when new recording is performed.)
3. Playback volume can be adjusted using the control provided on the back panel of the unit, next to the **"RECORD"** button.
4. Turn the unit off by rotating the **"VOLUME"** control knob located on the back of the unit counterclockwise until you hear an audible "click".

### **IMPORTANT NOTES:**

1. The memory of this unit is capable of holding up to 300 seconds of speech information total. This memory is divided in 8 equal segments of 37.5 seconds each. Segments are assigned to the buttons as shown below:

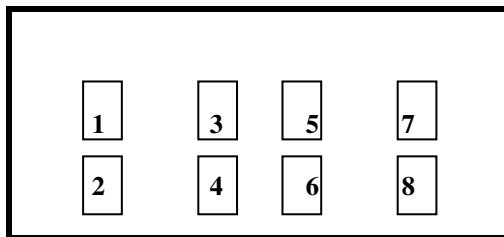

2. **Cheap Talk 8** will shut off automatically after playback to preserve battery life. However, device still drains a very small current in this "sleep" mode. Therefore, if the unit is not going to be used for a prolonged amount of time (more than a few days), it is recommended to disconnect the power by removing one of the batteries. Doing this can extend overall battery life.
3. If the unit fails to operate, or operates incorrectly, the following steps should be performed:
  - Disconnect the power by removing one of the batteries for 20 seconds to reset the unit. If the unit still does not work correctly after battery is reconnected, replace the batteries observing proper polarity.
  - Make sure there are no obstacles between the colored square buttons and the housing unit.

If you have any problems or questions about this unit or any of our products, please call our Technical Assistance Department at:

**1-800-TEC TOYS**

**(1-800-832-8697)**

[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)

**ENABLING DEVICES  
TOYS FOR SPECIAL CHILDREN, INC.  
385 WARBURTON AVENUE  
HASTINGS-ON-HUDSON,  
NEW YORK 10706**

**Cheap Talk 8  
(Black)**

**CATALOG # 1391-Bk**

**User's Guide**

**5-3-04**

**Toys for Special Children, Inc.**  
385 Warburton Avenue  
Hastings-on-Hudson, NY 10706  
[www.enablingdevices.com](http://www.enablingdevices.com)