

---

---

## Space Station (Slide Toy) #407

---

---

### USER'S GUIDE

---

**For Technical Support:**  
Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-TEC-TOYS or 1-800-832-8697  
customer\_support@enablingdevices.com

---

  
**Enabling Devices**

385 Warburton Avenue  
Hastings-on-Hudson, NY 10706  
Tel. 914.478.0960 / Fax 914.478.7030  
[www.enablingdevices.com](http://www.enablingdevices.com)

### *Blast Off!*

Activate your capability switch and watch as the spaceships race up and down and all around. Wonderful for improving visual tracking skills.

Size: 11"x 17" W. Requires 1 D Battery.

#### **Operation:**

1. Space Station requires 1 D battery (Not included). Use an alkaline battery only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly.
2. The battery compartment is located on the back of the unit. To remove the battery compartment cover, unscrew the small Phillips Head screw, then pull the cover open. Install a new battery, being careful to observe proper (+) & (-) battery polarity. Replace cover and screw.
3. Connect the unit to any external capability switch via the 1/8" cord jack, located on the side of the unit. If you need to use a 1/4" to 1/8" adapter for your switch, it must be a mono adapter, not stereo.
4. Make sure that the connections between the external capability switch and the adapter (if applicable) and the jack on the slide toy are tight. There should be no gap. Set the ON/OFF switch to OFF.
5. After assembling the track, load the supplied Space Ships into the bottom of the track. Make sure the slide toy is on a level table. Begin play by activating and maintaining activation of your external capability switch, this will start the slide toy picking up the Space Ships one by one. Once a Space Ship reaches the top of the track, the Space Ship then begins rolling down the track.
6. Once you release your switch the toy will stop. If you would like to the toy to run for a set amount of time, or by activation your switch once to start and then a second time to stop it use one of our switch latch timer modules (e.g.No.605) sold separately. There are also many of our switches that have a built in Latch/Timer function. Any of the switches in our catalog or website with a -X after the number has this feature.

**Important Note:** The Space Station will work normally with out the use of an external switch through its ON/OFF switch.

After use remove your switch (if applicable) and turn the ON/OFF switch OFF, this will prevent battery drain. If the Space Station is not going to be used for a long period of time, e.g. one month or longer, we recommend removing the battery from the unit during storage to prevent battery drain or leakage, which can cause damage to the Space Station.

#### **Troubleshooting:**

**Problem:** The slide toy does not work.

**Action #1:** Check or replace the D battery with a fresh new battery.

**Action #2:** Make sure the slide toy is on a flat level surface and that nothing is restricting any of the bears or track. Use only the Space Ships included with the slide toy, other slide toy pieces will not work with this product.

**Action #3:** Try a different switch that you know works with other toys/devices to rule out the slide toy as the source of the problem.

**Problem:** The slide toy runs non stop when I plug my switch into it.

**Action# 1:** Make sure your switch is plugged into the cord jack all the way, there should be no gap. If there is a gap the slide toy will run non stop.

#### **Care of Unit:**

The slide toy can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

**Do not submerge** the unit, as it will damage the contents and the electrical components.

**Do not use abrasive cleaners,** as they will scratch the surface of the unit.