

## Other Communication Devices

Communication Builder #7071

Communication Builder #7071-B (Black)

7 Level Blue Communication Builder #7075

7 Level Black Communication Builder #7075-B

On-the-Go 7-Level Communication Builder #7075G

4 Level Communication Builder #7077

6 Level-32 Message Communicator #3206

6-Level Communicator with Overlay #2391

6-Level Communicator with 1/8" Jacks and Overlay Option #2392

On-the-Go 6-Level Communicator #2391G

Icon Maker for Communication Builder #4021

---

### For Technical Support:

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-TEC-TOYS or 1-800-832-8697  
customer\_support@enablingdevices.com

---

---

## 7 Level Communication Builder

---

### Catalog # 7075 & 7075-B (Black)

### USER'S GUIDE

**Enabling Devices, Inc.**  
Hastings-on-Hudson, NY 10706  
[www.enablingdevices.com](http://www.enablingdevices.com)

The 7-Level Communication Builder allows the user to record and play back from 1, 2, 4, 8, or 16 different messages per level. There are seven levels for recordings, giving you a total of up to 112 messages (in the 16 window setting) at the touch of a button! Total record time is 300 seconds. The unit comes equipped with 5 sturdy frame/overlays in 1, 2, 4, 8 and 16 windows, which will retain your icon sheet (not included) and provide a separation between icon pads. **It is necessary to re-record all messages when you switch your windows.** The breakdown is as follows: Time per level by window setting, using all seven levels. (Picture overlays not included).

1 Window setting = 42 seconds per segment per level;

2 Window setting = 21 seconds per segment per level;

4 Window setting = 10½ seconds per segment per level;

8 Window setting = 5¼ seconds per segment per level;

16 Window setting = 2½ seconds per segment per level

Frame size is 8½" x 7". Size: 12½"L x 9¼"W x 4"H. Requires 4 AA Batteries

#### OPERATION:

1. Carefully turn the device over to reveal the battery compartment. A small Phillips head screw must be removed from the battery compartment cover in order to slide it off. Observing proper polarity, install four AA alkaline batteries. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together. Replace battery compartment cover and screw.
2. Place your icon grid over the gray message pad area then install the appropriate window frame. Next turn your window frame selector knob to the appropriate setting. Please note pre-made grid layouts can be made with our Easy To Use Icon Maker item No.4021 or Mayer-Johnson's Board maker program. Our Item No. 4008 for Macintosh computers and item No. 4009 for PC computers. If you have older versions of Boardmaker you can download the pre-made grids from [www.MayerJohnson.com](http://www.MayerJohnson.com).
3. Turn the unit on by turning the "ON/OFF/VOLUME" knob into the "ON" position. You will hear a slight audible click as you turn the knob.
4. The microphone is located on the bottom of the unit next to the red "RECORD" button. To record, depress and hold the "RECORD" button first, then depress one of the message pads simultaneously and speak into the microphone (your mouth should be approximately 4-6" from the microphone). Release both buttons once recording is over. Playback your message by pressing and releasing the same pad used to record. Each of the remaining message pads can be "programmed" in the same manner. (The unit will retain recorded information

indefinitely, even if the batteries are disconnected. Previously stored information will be erased only when a new recording is performed on the pad.)

5. Adjust playback volume by rotating the on/off "VOLUME" control knob to desired level. Turn the unit off by rotating the "VOLUME" control knob counterclockwise until you hear an audible "click". Using the "LEVEL" knob, select level 2. Repeat steps 2 and 3. Repeat procedures for remaining levels.

#### IMPORTANT NOTES:

**You must use the same window setting for all seven levels. (E.G.) you cannot use a four-window grid on level one and then go to level two and use a 16-window grid. If you did when you went back to level one, your messages would not play back properly.**

**This unit has a total record time of 300 seconds. The breakdown is as follows.**

#### Time per level by window setting.

- 1 Window setting = 42 seconds per segment per level
- 2 Window setting = 21 seconds per segment per level
- 4 Window setting = 10.5 seconds per segment per level
- 8 Window setting = 5.25 seconds per segment per level
- 16 Window setting = 2.5 seconds per segment per level

#### The 7 Level Communication Builder

- will shut off automatically after playback to preserve battery life. However, this device still drains a very small current in this "sleep" mode. Turn the power off when not in use. Doing this can extend overall battery life. Batteries should be removed during long time storage as they may leak and damage the unit.
- The unit may be wiped clean using mild detergent and water on a damp cloth. **Do not submerge the unit in water.**

#### TROUBLESHOOTING:

If the unit fails to operate, or operates incorrectly, try the following suggestions:

- ◆ Disconnect the power by removing all of the batteries for 20 seconds to reset the unit. If the unit still does not work correctly after batteries are reinstalled replace with new alkaline batteries. **Make sure proper polarity is observed or damage to unit may occur.**
- ◆ Make sure unit is on and volume is set at an audible level.
- ◆ Try to record again to see if the unit will "accept" the new recordings.