
Sip and Puff Switch – Double Closure #970

User's Guide

Enabling Devices/Toys for Special Children
385 Warburton Avenue
Hastings-on-Hudson, NY 10706
Tel. 914-478-0960 / Fax 914-478-7030
www.enablingdevices.com

Enabling Devices/Toys for Special Children
Hastings-on-Hudson, NY 10706
www.enablingdevices.com

The **Sip and Puff Switch – Double Closure** is used to activate two toys or devices with a single switch. Sipping on the tube will activate one toy/device, while puffing on the tube will activate a second toy/device. The switch is mounted on a 19” gooseneck and can be easily attached to most flat or tubular surfaces with the 3-way mounting clamp. Useful for when other possibilities for switch activation are exhausted. Extra straws can be ordered as well. Item # 960-S is a set of 20 straws. Lastly, the #970 is compatible with items #966-F and #977, the Anticontamination filters.

Operation:

1. This unit does not require any batteries to run.
2. Using the 3-way mounting clamp, secure the unit onto the wheelchair or bedrail or table, etc. Once it is clamped on, adjust the gooseneck so that the tube is positioned appropriately.
3. Attach the provided straw at the end of the tube simply by sliding it in.
4. Connect the Sip and Puff switch with one or two toys/devices using the male to male cords provided.
5. Insert the straw into your mouth. Sip to activate one toy/device or puff to activate the other toy/device. The toy/device will only remain activated as long as the sip or puff is sustained. Once you cease sipping or puffing the device/toy will go off.

Important Notes:

- This device requires a moderate strength sip or puff to operate.
- This switch is extremely sensitive and may remain in the “ON” position when the atmospheric pressure changes. If this occurs, reset the switch by separating the clear tubing where it is joined together by the white plastic connector, then reconnect.

Troubleshooting:

Problem: The unit fails to operate, or operates incorrectly.

Action #1: Make sure all of the connections are tight (between straws and tubing, between adapter and switch, between adapter and device/toy, etc.).

Action #2: Temporarily disconnect the clear tubing at the white connector to equalize the air pressure in the clear tubing, then reconnect.

Action #4: With time and use, the straw and/or the Anticontamination filter will become clogged. It is best to change these components before they become 100% clogged, as this could damage the unit. If possible, it is also a good idea to keep the switch box above the level of the user’s mouth. This limits the amount of saliva that enters the tubes.

Action #5: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Care of the Unit:

The Sip and Puff Switch – Double Closure can be wiped clean with any household multi-purpose, non-abrasive cleaner and disinfectant.

Change your filter and straw regularly.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.