

Some Other Popular Devices:

Pouring Cup #20

TV Disrupter #1545

Wheelchair Trainer #1520

Perceptual Motor Trainer #1130

Cup Detector #455

Biofeedback Weight Monitor #650

Enuresis Alarm #1090

Urine And Turn Alert #1095

Portable Therapist #1173

MotivAider #1153

Attendant Call Chime #1499

Battery Operated Scissors

Catalog # 9080

User's Guide

Enabling Devices & Toys for Special Children

385 Warburton Avenue

Hastings-on-Hudson, NY 10706

Telephone 914-478-0960 - Fax 914-478-7030

www.enablingdevices.com

Enabling Devices, Inc

385 Warburton Avenue

Hastings-on-Hudson, NY 10706

www.enablingdevices.com

6-27-05

Battery Operated Scissors

These easy-to-use switch-adapted scissors allow you to participate in art and other classroom, therapy or home activities. Just activate your switch and your scissors will easily cut through paper. Easy to remove from special mounting for those who have upper extremity use. Size: 6.5"L x 1.1/8" W x 2.1/4" H. Requires Capability Switch and two AA Batteries.

Operation:

1. To install the AA batteries, you must first remove the scissors from the base. Next carefully slide off the battery cover. Once the cover is off install the batteries paying close attention to proper (+) and (-) polarity. Please note there is one battery that will contact the small disk connected to the (-) battery post. The disk needs to sit flat against the bottom of that battery. The cord connected to the disk must be seated properly to close the cover and assure proper switch operation. Replace cover. Place the scissors back into the base.
2. Plug any capability switch into the cord jack at the end of the scissors; make sure it is a tight fit with no gaps.
4. Activate your switch and the scissors will run for as long as your switch is activated, once you release your switch the scissors will stop.
5. The scissors can be used with or without the base unit. For those without upper extremity use place the scissors into the base unit, then a therapist, teacher or parent can pass what you want to cut in front of the scissors, while the user activates his or her switch to start cutting.

Troubleshooting:

Problem: The scissors do not activate.

Action #1: Make sure that all batteries are installed following proper (+) and (-) polarity. Batteries must be fresh, if weak or dead replace.

Action #2: Make sure that the disk is seated flat against the (-) side of the battery it comes in contact with, as described in operation No.1.

Action #3: Make sure the connection between the scissors and switch are tight with no gaps.

Action #4: Check your switch with another device that you know works to rule out the scissors as the source of the problem.

Care of Unit:

The scissors can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697
Customer_Support@Enablingdevices.com