

Other Popular Toys:

Pouring Cup #20

Ring Around Bells #23

Dome Alone #69

Dome Alone with Switch #71

Drum #80

Musical Rolling Pin #160

Musical Light Box #200

Stacking Blocks #359

Stacking Ring Cone #360

Musical Top #400

Vibrating Light with Music #459

Glitter Roll #696

Motorized Glitter Roll #697

Butterfly Light and Music Puzzle #922

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697
customer_support@enablingdevices.com

Fido Switch - #18

USER'S GUIDE


Enabling Devices

385 Warburton Avenue
Hastings-on-Hudson, NY 10706
Tel. 914.478.0960 / Fax 914.478.7030
www.enablingdevices.com

By your request... Fun Animal Switches! You asked for 'em, you got 'em, irresistible, light touch animal switches. Bring some excitement into your classroom or home by using these colorful animal switches!

Operation:

1. The Fido Switch does *not* require any batteries to operate.
2. Plug the Fido Switch into the jack on your switch-adapted toy/device.
3. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
4. Make sure that the connection between the Fido Switch and your toy/device is tight. There should be no gaps.
5. Simply press anywhere on the Fido Switch, below the level of the nylon screws, to activate your toy/device.
6. The toy/device will only remain activated while the Fido Switch continues to be actuated. Once you release the switch, the toy/device will turn off.

Troubleshooting:

Problem: The Fido Switch does not activate your toy/device.

Action #1: Make sure that the Fido Switch is on a flat surface (not tilted or vertical). This provides for optimal functioning.

Action #2: Make sure that the connection between the Fido Switch and your toy/device is tight. There should be no gaps. This is a common error and an easy fix.

Action #3: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Action #4: Try a different adapter (if applicable) to rule this out as the source of the problem.

Care of Unit:

The Fido Switch can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.