
Sunny Sunshine Switch #892

USER'S GUIDE

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697
customer_support@enablingdevices.com


Enabling Devices

385 Warburton Avenue
Hastings-on-Hudson, NY 10706
Tel. 914.478.0960 / Fax 914.478.7030
www.enablingdevices.com

Children cannot resist this soft “sunny” switch/toy. Simply touch the happy yellow face or any of the 8 colorful plush arms to activate three music pieces, vibration and sparkling lights. This toy can also activate an external toy. Total continued playing time of the three 30-second music pieces is approximately 6 minutes. Size: 14"L x 14"W x 8"H. Requires 6 AA

Operation:

1. The Sunny Sunshine Switch requires 6 AA batteries to operate (Not included).
2. Gently turn the unit over to reveal the black battery compartments. Carefully remove the small screw from each of the battery compartment covers using a Phillips head screwdriver, and slide the battery compartment covers off. Observing polarity, install AA size batteries into each of the holders. Alkaline batteries must be used only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old, new or different types of batteries together. Replace the battery compartment covers and screws.
3. By simply pressing on any one of Sunny’s eight colorful arms, the user will be rewarded with a delightful display of lights, music and vibration. Sunny Sunshine can also be used as a switch to activate a switch-adapted toy or device. To do this, plug one end of the 1/8" to 1/8" cord (supplied) into the jack on the back of the Sunny Sunshine Switch and the other end into your switch-adapted toy or device. Be sure there is a tight connection. There should be no gaps. Then, press any one of Sunny’s eight colorful arms and the connected toy or device will activate for as long as the arm is depressed. When the arm is released the toy or device will stop.

Please Note: The lights and music will play for approximately 6 minutes, if activation is maintained. To continue playing music beyond 6 minutes, the toy must be reactivated by repressing one of the arms.

Troubleshooting:

Problem: The unit fails to operate, or operates incorrectly.

Action #1: Disconnect the power by removing one of the batteries for 20 seconds in order to reset the unit. Re-install the batteries. If the unit does not work correctly, replace all the batteries paying careful attention to proper polarity.

Action #2: Make sure that there no objects or debris blocking or restriction the switch plate.

Problem: The unit works fine as a switch, but will not light up or vibrate.

Action #1: Check your batteries – There is a 99% chance that either one is installed incorrectly and/or one is weak/dead.

Action #2: If lights and music work but no vibration, check the batteries in the two AA compartment. Replace if weak or dead.

Problem: The lights and or vibration are weak.

Action: Check your batteries, there is a good change the batteries are weak or of the wrong type like heavy duty or rechargeable. Change with new alkaline batteries and never mix old and new or types of batteries together.

Care of the Unit:

The Sunny Sunshine Switch can be wiped clean with any mild household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.