

Our new Round Cheap Talk 4 is easy to mount on wheelchair trays, bedsides and tabletops with a convenient gooseneck clamp set-up. Record 4 messages on 12 levels with a total recording time of 300 seconds. The Round Cheap Talk 4 is great for providing communication and cognitive support, as well as enhancing expressive and receptive language skills. The icon size is 2 x 2. Size: 8 3/4" Diameter x 2"D. Requires 4 AA Batteries.

OPERATION:

1. Carefully turn over unit to reveal battery compartment. Use a screwdriver to remove the screw on the battery compartment cover and slide the cover off. Observing polarity, install four AA batteries (Not included) into the holder. Screw the battery compartment cover back on. Turn the **ON/OFF/VOLUME** knob clockwise to **ON**.
2. The microphone and red **"RECORD"** button are located on the back panel of the unit. To record, first depress and hold the **"RECORD"** button, then while holding the **"RECORD"** button depress one of the 4 colored squares and speak into the microphone. Release both buttons once recording is over. Playback your message by pressing and releasing the same square used to record. Each of the remaining buttons can be "programmed" in the same manner. Once you have recorded all four messages on level one, continue on to record messages on all levels using the same method. (The unit will retain recorded information indefinitely, even if the batteries are disconnected. Previously stored information will be erased only when new recordings are performed.)
3. Change levels by turning the **"LEVEL"** control knob provided on the back panel of the unit to the desired level.
4. Playback volume can be adjusted using the **"VOLUME"** control knob provided on the back panel of the unit, next to the **"RECORD"** button.
5. Turn the unit off by rotating the **"VOLUME"** control knob located on the back panel of the unit counterclockwise until you hear an audible "click".

6. The four large, colorful plates provide ample space for words, symbols or pictures. These visuals should be cut or designed to fit in the square inset area of each plate, and then covered with the clear overlays provided. To do this, simply bend the clear plastic overlay and slip the sides into the slots.

Mounting Information:

1. Using the universal mounting clamp, secure the unit onto a wheelchair, bedrail or table, etc. Once it is clamped on, adjust the gooseneck so that the Cheap Talk is positioned appropriately for the user.
2. If desired the Cheap Talk can be removed from the Gooseneck arm, by un-screwing it from the arm. The Cheap Talk can now be used on a table top or wheelchair tray like our standard Cheap Talk models.

IMPORTANT NOTES:

1. The memory of this unit is capable of holding up to 300 seconds of speech information total. This provides for 6 seconds of recording time per square per level. No bleed over of messages will occur because of our time-lock feature.
2. If the unit fails to operate, or operates incorrectly, the following steps should be performed:
 - Disconnect the power by removing one of the batteries for 20 seconds to reset the unit. If the unit still does not work correctly after battery is reconnected, replace the batteries observing proper polarity.
 - Make sure there are no obstacles between the colored square buttons and the housing unit.

If you have any problems or questions about this unit or any of our products, please call our Technical Assistance Department at:

(1-800-832-8697)

customer_support@enablingdevices.com

Other Communication Devices

Communication Builder #7071

Communication Builder #7071-B (Black)

7 Level Blue Communication Builder #7075

7 Level Black Communication Builder #7075-B

1 Level-32 Message Communicator #3201

2 Level-32 Message Communicator #3202

3 Level-32 Message Communicator #3203

6 Level-32 Message Communicator #3206

Auditory Queuing Communicator 16 level #4399

2 Level Communicator with Overlay Option #2302

2 Level Communicator with 1/8" Jacks And Overlay Option #2303-8

6-Level Communicator with Overlay #2391

6-Level Communicator with 1/8" Jacks and Overlay Option #2392-8

ENABLING DEVICES, INC.

385 Warburton Avenue

Hastings-on-Hudson, NY 10706

(914) 478-0960

Round Cheap Talk 4 With 12 Levels

Cat. No. 1336-R

User's Guide

5-31-05