
Foxy Fox #954

USER'S GUIDE

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697
customer_support@enablingdevices.com


Enabling Devices

385 Warburton Avenue
Hastings-on-Hudson, NY 10706
Tel. 914.478.0960 / Fax 914.478.7030
www.enablingdevices.com

See Foxy trot!

Add your capability switch and watch Foxy shake her head and tail, walk forward, and speak! She is adorable and so much fun. Size: 10"L x 9"W x 10"H. Requires 2C Batteries

Operation:

1. Foxy Fox requires 2 C Batteries to operate. Alkaline batteries must be used (e.g. Duracell or Energizer Brands). Do not use rechargeable batteries because they supply a lower voltage and the unit may not perform as well. Never mix old and new batteries together or different brands or types together.
2. Turn Foxy Fox upside down to reveal the battery compartment. Open battery compartment and install 2 C batteries paying close attention to proper battery polarity. Close the battery compartment.
3. Connect Foxy Fox to any external capability switch (not included) via the jack located at the end of Foxy Fox's cord. Foxy Fox is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure to have a tight connection. There should be no gaps.
4. Actuate your external capability switch and Foxy Fox will spring to life! She will shake her head and tail, walk forward, and speak!
5. Foxy Fox will remain on while the external capability switch continues to be actuated. Once you release the external capability switch, there will be a brief delay, and then Foxy Fox will turn off.

Troubleshooting:

Problem: Foxy Fox does not function correctly.

Action #1: Replace the batteries paying close attention to proper (+) & (-) battery polarity.

Action #2: Make sure that the connection between the external capability switch and the jack is TIGHT. There should be NO gaps.

Action #3: Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

Care of Unit:

- Do not inhibit Foxy Fox's movement, as this may cause irreversible damage.
- Do not lift Foxy Fox up by her head, tail or legs, or drop Foxy Fox.
- If you will not be using Foxy Fox for an extended amount of time, we recommend removing the batteries.

Do not submerge the unit in water, as it will damage the electrical components. Foxy Fox is surface washable only.