
Rolli Pop #161

USER'S GUIDE

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697
customer_support@enablingdevices.com



Enabling Devices

385 Warburton Avenue
Hastings-on-Hudson, NY 10706
Tel. 914.478.0960 / Fax 914.478.7030
www.enablingdevices.com

Activate your capability switch and the cup scoops up a ball and drops it out on top. The ball then rolls down the ramp at a perfect speed for increasing visual attention and tracking. Music plays while switch is activated. Size: 25"L x 12"W x 23"H. Requires 4 AA Batteries.

Operation:

1. The Rolli Pop requires four AA batteries (Not included). Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. The battery compartments are located on the side of the unit. To remove the battery compartment covers, unscrew the small Phillips Head screws, then slide the covers back and then off. Install new batteries, being careful to observe proper (+) & (-) battery polarity. Replace cover and screw.
3. Connect the unit to any external capability switch via the 1/8" jack, located on the side of the unit. If you need to use a 1/4" to 1/8" adapter for your switch, it must be a mono adapter, not stereo.
4. Make sure that the connections between the external capability switch and the adapter (if applicable) and the jack on the Rolli Pop are tight. There should be no gaps.
5. Load the supplied balls into the ball tray on the base of the unit. Make sure the Rolli Pop is on a level table. Begin play by activating and maintaining activation of your external capability switch, this will start the scooper picking the balls up one by one and playing a children's well known musical tune. Once a ball reaches the top of the belt, it begins rolling down the track.

6. Once you release your switch the music and balls will stop. If you would like to the toy to run for a set amount of time, or by activation your switch once to start and then a second time to stop it use one of our switch latch timer modules (e.g.No.605) sold separately. There are also many of our switches that have a built in Latch/Timer function. Any of the switches in our catalog or website with a -X after the number has this feature.

Troubleshooting:

Problem: The Rolli Pop does not work.

Action #1: Check or replace the four AA batteries with fresh new batteries. Never mix different brands or old and new batteries.

Action #2: Make sure the Rolli Pop is on a flat level surface and that nothing is restricting any of the balls or conveyer belt. Use only the balls included with Rolli Pop, other balls will not work with this product.

Action #3: Try a different switch that you know works with other toys/devices to rule out the Rolli Pop as the source of the problem.

Care of Unit:

The Rolli Pop can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.