

---

---

## Frog Switch - #17

---

---

## USER'S GUIDE

---

**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-TEC-TOYS or 1-800-832-8697  
[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)

---



**Enabling Devices**

50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

By your request.... Fun Animal Switches!

You asked for 'em, you got 'em, irresistible, light touch animal switches. Bring some excitement into your classroom or home by using these colorful animal switches!

**Operation:**

1. The Frog Switch does *not* require any batteries to operate.
2. Plug the Frog Switch into the jack on your switch-adapted toy/device.
3. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
4. Make sure that the connection between the Frog Switch and your toy/device is tight. There should be no gaps.
5. Simply press anywhere on the Frog Switch, below the level of the nylon screws, to activate your toy/device.
6. The toy/device will only remain activated while the Frog Switch continues to be actuated. Once you release the switch, the toy/device will turn off.

**Troubleshooting:**

**Problem:** The Frog Switch does not activate your toy/device.

**Action #1:** Make sure that the Frog Switch is on a flat surface (not tilted or vertical). This provides for optimal functioning.

**Action #2:** Make sure that the connection between the Frog Switch and your toy/device is tight. There should be no gaps. This is a common error and an easy fix.

**Action #3:** Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

**Action #4:** Try a different adapter (if applicable) to rule this out as the source of the problem.

**Care of Unit:**

The Frog Switch can be wiped clean with any household multi-purpose cleaner and disinfectant.

**Do not submerge the unit**, as it will damage the contents and the electrical components.

**Do not use abrasive cleaners**, as they will scratch the surface of the unit.