

Other Popular Similar Devices:

Take N' Talk Series #8701-8704

Take or Place N' Talk #8705

Cheap Talk 4 Inline – Direct #3054

Communication Builder #7071

Cheap Talk 8 – Direct #1391

Hip Talk Plus #5012

Four Compartment Communicator with Speech and Lights
#1239

Go! Board #3334, #3336, #3338

Go! Board Activity Station #3500

For Technical Support:

Call our Technical Service Department,
Monday through Friday, 9am to 5pm (EST).
1-800-TEC-TOYS or 1-800-832-8697
customer_support@enablingdevices.com

Upright Take N' Talks

Catalog

8709 Upright Take N' Talk Four

8709-L Upright Take N' Talk 12-Levels

8709-A Upright Take N' Talk 12-Levels

(Age Appropriate)

User's Guide

Enabling Devices/Toys for Special Children

385 Warburton Avenue

Hastings-on-Hudson, NY 10706

Telephone 914-478-0960 – Fax 914-478-7030

www.enablingdevices.com

Upright Take N' Talks

Clamp it!

Taking or replacing the Velcro mounted icon holders will play your pre-recorded message. Also available with 12 Levels in an age appropriate version. Clamp sold separately, see below. Size: 13.25"L x 4.5"W x 3.5"H. Requires four AA Batteries (Not included).

Operation:

1. Carefully turn unit over and remove the small screw from the battery compartment. Slide the battery compartment cover off. Observing proper polarity, install four AA batteries (included) into the holder. Alkaline batteries are recommended because they last longer. Do not use rechargeable batteries because they supply lower voltage and the unit may not perform as well. Replace the battery compartment cover and replace the screw.
2. Turn the unit on by turning the ON/OFF/VOLUME knob (black knob) clockwise. It will click into the on position. Make sure to turn it to an audible volume level.
3. You may hear some of the "test recordings." This is all right. It is due to the light hitting the photocells. Do not worry.
4. It is very important that the photocells (the small dot in the middle of each compartment) be covered while trying to record. Place your icon squares over the photocells before making any recordings. To record, simply press and hold the "RECORD" button first (labeled "REC") then simultaneously holding down one of the black buttons next to it. While continuing to hold the buttons down, speak your message. You should speak in a normal volume with your mouth approximately 6-8 inches from the microphone (the small black felt-like circle). Release both buttons when you finish recording. Record on the other buttons in the same way. You have 5 seconds for each recording so plan accordingly. **Models 8709-L and A have 12 levels, this allows you to record 12 set of four message choices for a total of 48 messages, at six seconds per message.** To use the levels turn the level knob to level one and record your messages as previously noted. To record the next level turn the level knob to level two. Record as before and record remaining levels the same way.
5. Once you have finished recording, turn the unit back over. Now place your pictures, symbols, icons, etc. into the squares, by first removing the clear Mylar cover. Once your icon is in place replace the clear cover. You can choose to either have the recordings play when you TAKE an icon holder from the compartment OR when you PLACE an icon holder in the compartment. The toggle switch controls this. One side is for TAKING and the flip side is for PLACING.
6. **If you chose to have the recordings play when you TAKE the object:** When you remove the picture, symbol, item, etc. from the compartment, your message will play back to you. If you remove more than one item at a time, the first photocell exposed to light will be the only one that plays.
If you chose to have the recordings play when you PLACE the object: When you place the picture, symbol, item, etc. in the compartment, your message will play back to you. If you place more than one square at a time, the first photocell cut off from the light supply will be the only one that plays.
7. You can then place the square back into the compartment, or remove it from the compartment again, if you choose.
8. The unit will retain recorded information indefinitely. Previously stored information will be erased only when a new recording is performed.

9. Playback volume can be adjusted using the "ON/OFF/VOLUME" control knob (black knob).

Important Notes:

- The memory of this unit is capable of holding up to a total of 20 seconds of speech information. This allows for 5 seconds per compartment. Please plan accordingly.
- Subdued/dim lighting may cause erratic operation. Normal room lighting is best. DO NOT leave in direct sunlight, as this will irreversibly damage the photocells and void out your warranty.
- Please take caution not to spill anything on the photocells, or expose the photocells to any liquid. When cleaning the unit, wipe around the photocells but do not actually wipe it.
- Do not put excessive pressure on the "REC" button or any of the black buttons.

Optional C-Clamp mounting:

All of our upright Take N' Talks can be clamped to a table top or lap tray with the use of our C-Clamp set No.2321 (Sold separately). On each side of the unit there is a clamping area. Caution tighten the clamps only enough to hold the unit securely. Over-tightening of the clamps can damage the Take N' Talk which will void your warranty.

Troubleshooting:

Problem: The unit fails to operate, or operates incorrectly.

Action #1: Make sure that you are using fresh alkaline batteries and that they are installed according to proper battery polarity.

Action #2: Make sure that the unit is turned up to an audible volume level. If it does seem to be up, you can try to "re-set" the unit by turning it off, waiting about 20 seconds, and then turning it back on.

Action #3: Make sure that the photocells have not been exposed to any liquids and/or direct sunlight.

Problem: The unit plays back the messages, but does not record new messages.

Action #1: Make sure that the batteries are installed according to proper polarity.

Action #2: Make sure that the microphone is not covered up while you are trying to record.

Problem: The sound quality of the recordings is not good.

Action #1: Make sure that you are recording using a normal speaking volume and your mouth is about 6-8" from the microphone. Speaking loudly will cause the sound to resonate.

Problem: The messages are playing erratically.

Action #1: Make sure that the pictures, symbols, items, etc. that you are using are (a) not clear, translucent or white, and (b) that they completely cover the photocell.

Action #2: Make sure that you have not exceeded the 5 seconds per compartment limit. If you do go over the 5 seconds, the messages will bleed over into each other and cause confusion. If this happens, it is best to start over again with recording, as opposed to trying to "fix" the bleed over.

Care of the Unit:

The Take or Place N' Talks can be wiped clean with any household multi-purpose, non-abrasive cleaner and disinfectant. DO NOT CLEAN THE PHOTOCELLS – WIPE AROUND THEM!

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.