
Puss 'N Boots #3918

USER'S GUIDE

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697
customer_support@enablingdevices.com



Enabling Devices

385 Warburton Avenue
Hastings-on-Hudson, NY 10706
Tel. 914.478.0960 / Fax 914.478.7030
www.enablingdevices.com

Puss 'n Boots is slick and this plush cat will do his best to let you know how wonderful he is. He will sweet-talk you when you approach him and will invite you to hug, pet, and cuddle him when you activate your capability switch. Size: 14 1/2 "H x 9 1/2"W x 8"L. Requires Capability Switch and 3 AA Batteries.

Operation:

1. Puss 'N Boots requires 3 AA Batteries to operate. Alkaline batteries must be used (e.g. Duracell or Energizer Brands). Do not use rechargeable batteries because they supply a lower voltage and the unit may not perform as well. Never mix old and new batteries together or different brands or types together.
2. The battery compartment is located in Puss 'N Boot's back. Open the Velcro closure. Next using a Phillips head screwdriver remove the screw and install 3 AA batteries paying close attention to proper (+) and (-) battery polarity. Replace the battery compartment cover and secure it with the screw. Set the ON/OFF/TRY ME switch to ON.
3. Connect Puss 'N Boots to any external capability switch (not included) via the jack located at the end of Puss 'N Boot's cord jack. Puss 'N Boots is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure you have a tight connection. There should be no gaps.
4. Actuate your external capability switch and Puss 'N Boots will come to life! He will speak different phrases to you.
5. You can also play with Puss 'N Boots without an external capability switch. Make movement in front of his eyes or pet his head and he will purr for you.
6. When using Puss 'N Boots with or without a switch take note; that his eyes detect change in light in the room or when you make movements in front of him. Very dim or brightly lit rooms may cause false activations. Movement in front of him may also trigger activation.

Please Note: Some of Puss 'N Boots's original functions noted in the manufactures instructions may not work or are random due to switch adaptation.

Troubleshooting:

Problem: Puss 'N Boots does not function correctly.

Action #1: Replace the batteries paying close attention to proper (+) and (-) battery polarity.

Action #2: Make sure that the connection between the external capability switch and the jack is TIGHT. There should be NO gaps.

Action #3: Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

Care of Unit:

If you will not be using Puss 'N Boots for an extended amount of time, we recommend removing the batteries.

Do not submerge the unit in water, as it will damage the electrical components. Puss 'N Boots is surface washable only.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.