
Baby Dino #3921

USER'S GUIDE

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697
customer_support@enablingdevices.com



Enabling Devices

385 Warburton Avenue
Hastings-on-Hudson, NY 10706
Tel. 914.478.0960 / Fax 914.478.7030
www.enablingdevices.com

Roarrrrrrr!

Baby Triceratops is ready to roam the earth. Activate your capability switch and he will move his head and walk forward making roaring sounds. Size: 11 3/4"L x 4 1/2"W x 5"H. Requires Capability Switch and 2 AA Batteries.

Operation:

1. Baby Dino requires 2 AA Batteries to operate. Alkaline batteries must be used (e.g. Duracell or Energizer Brands). Do not use rechargeable batteries because they supply a lower voltage and the unit may not perform as well. Never mix old and new batteries together or different brands or types together.
2. Turn Baby Dino upside down to reveal the battery compartment. Remove the small screw with a Phillips Head screwdriver. Install 2 AA batteries paying close attention to proper battery polarity. Replace the battery compartment cover and screw.
3. Connect Baby Dino to any external capability switch (not included) via the jack located at the end of Baby Dino's cord. Baby Dino is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure to make a tight connection. There should be no gaps.
4. Actuate your external capability switch and Baby Dino will come to life! He will walk forward, move his head and makes sounds.
5. Baby Dino will remain on while the external capability switch continues to be actuated. Once you release the external capability switch, Baby Dino will turn off.

Troubleshooting:

Problem: Baby Dino does not function correctly.

Action #1: Replace the batteries paying close attention to proper (+) & (-) battery polarity.

Action #2: Make sure that the connection between the external capability switch and the jack is TIGHT. There should be NO gaps.

Action #3: Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

Care of Unit:

- Do not inhibit Baby Dino's movement, as this may cause irreversible damage.
- Do not use Baby Dino on heavy plush type carpeting.
- Do not lift Baby Dino up by his head, tail or legs, or drop Baby Dino.
- If you will not be using Baby Dino for an extended amount of time, we recommend removing the batteries.

Do not submerge the unit in water, as it will damage the electrical components. Baby Dino is surface washable only.