

Hip-Step Talking Sequencer has a 300-second maximum message capacity and comes in a soft nylon zipper pack to be worn around the waist. It also features large switchplates on top for easy access. This communicator expresses a series of your pre-recorded messages (300 second limit) when the switch plate is activated. **Hip-Step Talking Sequencer** can be used for a number of different purposes:

- Story-telling
- Sequencing
- Testing
- Medical Instructions
- Multiplication
- Memorization

OPERATION:

1. Carefully turn over unit to reveal battery compartment. Install 4 AA batteries (included), observing proper polarity (+/-), in battery holder.
2. The **RECORD** switch is located on the bottom of the unit and the microphone is on the back of the unit. To record, first depress and hold the **RECORD** button then press the **PLAY/RECORD** switch plate and speak the first message into the microphone. When you are finished recording the first message, release only the **PLAY/REC** plate. To record all subsequent messages, simply depress **PLAY/RECORD** switch plate and speak into the microphone, releasing only the **PLAY/RECORD** plate after each message. When finished with all your messages then release both **REC** button and **PLAY/REC** switch plate.
3. Messages can vary in length and the total record time is 300 seconds.
4. To play the messages, simply depress the **PLAY/RECORD** switchplate. Activation of the switchplate will play the first message; the second activation will play the second message, and so on. To interrupt the messages and start at the beginning of the sequence, press the **RESET** switchplate.
5. The unit will retain recorded information indefinitely, even if the battery is disconnected. Previously stored information will be erased only when new recording is performed.
6. *Please Note:* When re-recording messages, all messages on the unit will be erased.

IMPORTANT NOTES:

The memory of this unit is capable of holding up to 300 seconds of speech information total.

The **VOLUME** control knob is located on the bottom of the unit. Turn clockwise to increase the volume, counterclockwise to decrease the volume.


If the unit fails to operate, or performs incorrectly, the following steps should be performed:

- Disconnect the batteries for 30 minutes to reset the unit. If the unit still does not work after battery is reconnected, replace the batteries with new ones.
- Make sure there are no obstacles between the switchplates and the housing of the unit.

If you have any problems or questions about this unit or any of our products, please call our Technical Assistance Department at:

(1-800-832-8697)

customer_support@enablingdevices.com

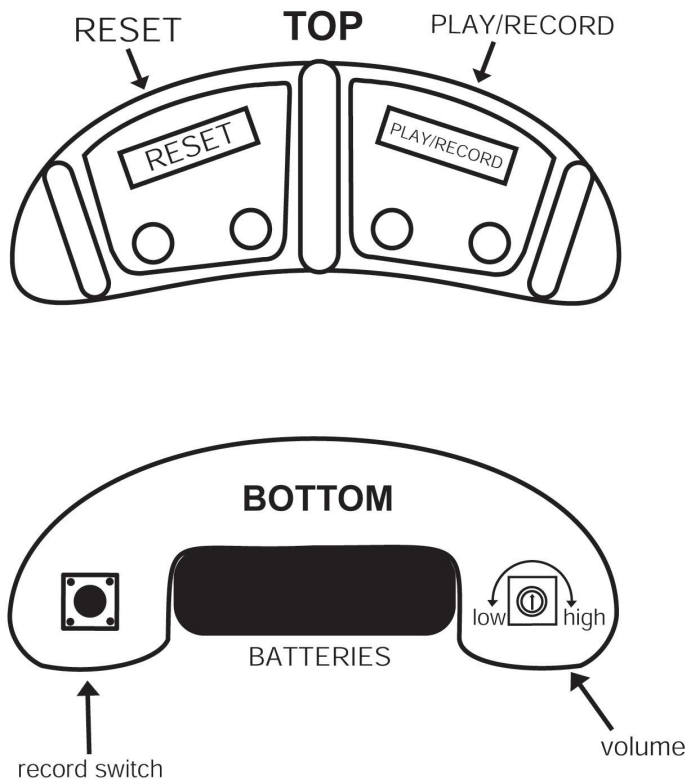

ENABLING DEVICES, INC.
385 Warburton Avenue
Hastings-on-Hudson, NY 10706
www.enablingdevices.com

Hip Step Talking Sequencer

Catalog No.
5075

User's Guide