

**Other Popular Toys:**

Pouring Cup #20

Ring Around Bells #23

Dome Alone #69

Dome Alone with Switch #71

Drum #80

Musical Rolling Pin #160

Musical Light Box #200

Stacking Blocks #359

Stacking Ring Cone #360

Musical Top #400

Vibrating Light with Music #459

Glitter Roll #696

Motorized Glitter Roll #697

Butterfly Light and Music Puzzle #922

---

**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-TEC-TOYS or 1-800-832-8697  
customer\_support@enablingdevices.com

---

---

---

**The Happy Switch - #14**

---

---

**USER'S GUIDE**

  
**Enabling Devices**

385 Warburton Avenue  
Hastings-on-Hudson, NY 10706  
Tel. 914.478.0960 / Fax 914.478.7030  
[www.enablingdevices.com](http://www.enablingdevices.com)

## **A switch To Perk up YOUR DAY!**

**The latest edition to our assortment of light touch switches is FUN for all ages! It will brighten up your day, so add this one to your collection today! Comes with a 1/8" plug.**

### **Operation:**

1. The Happy Switch does not require any batteries to operate.
2. Plug the Happy Switch into the jack on your switch-adapted toy/device.
3. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
4. Make sure that the connection between the Happy Switch and your toy/device is tight. There should be no gaps.
5. Simply press anywhere on the Happy Switch to activate your toy/device.
6. The toy/device will only remain activated while the Happy Switch continues to be actuated. Once you release the switch, the toy/device will turn off.

### **Troubleshooting:**

**Problem:** The Happy Switch does not activate your toy/device.

**Action #1:** Make sure that the Happy Switch is on a flat surface (not tilted or vertical). This provides for optimal functioning.

**Action #2:** Make sure that the connection between the Happy Switch and your toy/device is tight. There should be no gaps. This is a common error and an easy fix.

**Action #3:** Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

### **Care of Unit:**

The **Happy Switch** can be wiped clean with any household multi-purpose cleaner and disinfectant.

**Do not submerge** the unit, as it will damage the contents and the electrical components.

**Do not use abrasive cleaners**, as they will scratch the surface of the unit.