
Happy Cow - #3063

USER'S GUIDE

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697
customer_support@enablingdevices.com



Enabling Devices

385 Warburton Avenue
Hastings on Hudson, NY 10706
Tel. 914.478.0960 / Fax 914.478.7030
www.enablingdevices.com

Mooooo!

Activate your capability switch and he will laugh and bounce up and down for joy. He is so much fun to play with. Also works by squeezing his hoof. Size: 12"H. Requires 4 AA Batteries.

Operation:

1. Happy Cow requires 4 AA batteries (Not included) to operate. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. The battery compartment is located in Happy Cow's back. Open the Velcro flap then, depress the battery cover latch and pull the cover off. Next install 4 AA batteries paying close attention to proper battery polarity.
3. Replace the battery compartment cover. Turn the ON/OFF switch to ON, which is located the above the battery compartment.
4. Connect Happy Cow to any external capability switch (not included) via the jack located at the end of his cord. Happy Cow is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure you have a tight connection. There should be no gaps.
5. Actuate your external capability switch and Happy Cow will come to life! When you activate your switch or squeeze his hoof, he gives a few contented moos. Then comes the big surprise! He suddenly begins shaking, spinning and bellowing maniacally. Better head for the hills before you get stamped.

6. When you are finished using Happy Cow turn his ON/OFF switch to the OFF position.

Troubleshooting:

Problem: Happy Cow does not function correctly.

Action #1: Replace the batteries paying close attention to proper battery polarity.

Action #2: Make sure that the connection between the external capability switch and the jack is TIGHT. There should be NO gaps.

Action #3: Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

Care of Unit:

If you will not be using Happy Cow for an extended amount of time, we recommend removing the batteries.

Do not submerge the unit in water, as it will damage the electrical components. Happy Cow is surface washable only.