
Call Light on a Gooseneck #1498

USER'S GUIDE

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697
customer_support@enablingdevices.com


Enabling Devices

385 Warburton Avenue
Hastings-on-Hudson, NY 10706
Tel. 914.478.0960 / Fax 914.478.7030
www.enablingdevices.com

Mounts anywhere!

A wonderful way to get your teacher's attention when you need help. Mount anywhere and activate your capability switch and the light remains on until your teacher switches it off. Size: 33 1/2"H. Requires 2 AAA Batteries.

Operation:

1. Call Light requires 2 AAA batteries (Not included) to operate. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. The battery compartment is on the back of the Control Box, located at the base of the Gooseneck. Slide the cover off the battery compartment. Observing (+) & (-) polarity, install the two batteries into the holder. Replace the battery compartment cover.
3. Using the mounting clamp, attach the Call Light to a Wheel Chair, Table or Desk. Tighten the clamp enough to hold the Call Light securely, but do not over-Tighten the clamp.
4. To Activate the Call Light, first connect your external Capability switch via the 1/8" jack on the Control Box. Make sure you have a tight connection. There should be no gaps.
5. Activate your Capability switch once will turn the Call Light ON. To turn the Call Light OFF, the Teacher/Care Giver has to press the OFF/RESET button on the top of the Control Box.

Troubleshooting:

Problem: The unit fails to operate, or operates incorrectly.

Action #1: Make sure that the connection between the external capability switch and the Call Light is TIGHT. There should be NO gaps.

Action #2: Check that all the batteries are installed, following proper (+) & (-) polarity. Also check that all batteries are fresh.

Action #3: Replace the batteries paying close attention to proper (+) and (-) battery polarity.

Action #4: Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

Care of Unit:

If you will not be using the Call Light for an extended amount of time, we recommend removing the batteries.

Call Light can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.