
Spinning 1st Friends Switch #891

USER'S GUIDE

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697
customer_support@enablingdevices.com


Enabling Devices

385 Warburton Avenue
Hastings-on-Hudson, NY 10706
Tel. 914.478.0960 / Fax 914.478.7030
www.enablingdevices.com

As a toy alone or as a switch, this delightful spinning toy will transform any child's world into a sensory playground. The slightest touch will activate a toy or flood the user with music, dancing lights and vibration. Size: 10"L x 5"W x 7"H. Requires 4 AA Batteries.

Operation:

1. The Spinning 1st Friends Switch requires four AA batteries (Not included). The battery compartments are located under the base of the unit. Carefully turn the unit over, and then remove the Philips head screws from the battery covers. Install new batteries, being careful to observe proper battery polarity. Alkaline batteries must be used only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old, new or different types of batteries together. Replace the battery compartment covers and screws.
2. Rotate or spin the Spinning 1st Friends ferries wheel and you will be rewarded with lights, music and vibration. Stop rotating or spinning of the Ferries Wheel will cause the Lights, Music and Vibration to stop.
3. To use the Spinning 1st Friends Switch as a switch plug one end of the supplied jack cord into the Spinning 1st Friends Switch's female jack and the other end into your adapted toy/device. Rotate or spin the Ferries Wheel and your toy/device will activate as long as rotating or spinning of the Ferries Wheel is maintained.
4. **Important Notes:** when used as a switch the Spinning 1st Friends Switch's Lights, Music & Vibration will be deactivated. You cannot use a switch to activate the Spinning 1st Friends Switch; it will only work as a sensory toy or a switch.

Troubleshooting:

Problem: The Spinning 1st Friends Switch has no lights, Music or Vibration.

Action #1: Check all batteries for strength and for proper battery polarity. Replace if weak or dead.

Action #2: Make sure the supplied 1/8" to 1/8" cord is not plugged into the Spinning 1st Friends Switch. This will disable the Light, Music & Vibration even if it is not plugged into another toy or device.

Problem: The Spinning 1st Friends Switch does not work at all.

Action: Check all Batteries for strength and for proper battery polarity. Replace if weak or dead.

Problem: Spinning 1st Friends Switch does not activate a toy or device.

Action # 1: Make sure all connections are tight.

Action # 2: Check batteries in toy/device, replace if weak or dead.

Action # 3: If adapters are being used they must be Mono, stereo adapters will not work.

Action #4: Try your toy or device with another switch you know works to rule it out as the source of the problem.

Care of the Unit:

The Spinning 1st Friends Switch can be wiped clean with any mild household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.