

Care of Unit:

The **6-Level Communicator** can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit as it will damage the electrical components.

Do not use abrasive cleaners as they will scratch the surface of the unit.

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697

6-Level Communicator

The **6-Level Communicator** is a self-contained communication device which allows the user to record and play back 48 different messages of up to five seconds each. Interchangeable picture cards (yours or ours) slide behind the switch plate face. The user simply touches one of the 2-inch square pictures to activate the corresponding message. A switch under the unit allows you to select the set of pre-recorded messages for each card. Excellent for use as a communication device and a teaching tool.

Operation:

1. Carefully turn over unit to reveal battery compartment. Remove the cover then, observing polarity, install four AA size batteries (included) into the holder. Alkaline batteries are recommended because they last longer. Do not use rechargeable batteries because they supply lower voltage and the unit may not perform as well.
 2. Move the **LEVEL SELECTOR** Switch located under the unit to Level 1.
 3. To record, depress and hold the red **RECORD** button located under the unit. At the same time, depress one of the eight square plates on the front of the unit and speak into the microphone located on the back panel of the unit.
-

Operation (continued):

4. Release both the plate and the recording button when the recording is over. Play back your message by pressing and releasing the same square used to record. Each of the remaining squares can be programmed in the same manner. The unit will retain recorded information indefinitely, even if the battery is disconnected. Previously stored information will be erased only when a new recording is performed.
5. Move the **LEVEL SELECTOR** Switch to Level 2 and repeat Step 3. Continue programming additional levels, if needed.
6. Draw or print corresponding icons or words on a sheet of paper measuring 5-1/2" x 10" so they will appear through the eight cutouts on the housing, when inserted.
7. Carefully insert the sheet between the two top cover plates. Slide the page down until your icons or words appear through the eight square windows.
8. To playback, select the level and then touch the appropriate window.
9. Playback volume can be adjusted by turning the **VOLUME CONTROL** knob located under the unit.
10. Turn the unit off by rotating the **VOLUME CONTROL** knob counterclockwise until you hear an audible "click".

Important Notes:

When switching from one level to another, allow 5 seconds for the unit to switch programs, before recording new messages or playing back existing messages.

Additional paper inserts may be cut from regular paper to the recommended size of 5-1/2" x 10" to fit properly into the housing.

The **6-Level Communicator** will shut off automatically after playback, to preserve battery life. However, the device still drains a very small current in this "sleep" mode. Therefore, if the unit is not going to be used for a prolonged amount of time (more than a few days), it is recommended that the unit be turned off, to extend overall battery life. This can be done by turning the **VOLUME CONTROL** knob counterclockwise, until you hear an audible "click".

Troubleshooting:

Problem: The unit fails to operate, or operates incorrectly.

Action #1: Disconnect the power by turning OFF the unit for 20 seconds to reset the unit. If the unit still does not work correctly after the power is reconnected, replace the batteries observing proper polarity.

Action #2: If the unit still does not work, make sure there is no dirt or obstacles between the square plates and the housing of the unit.

Some Other Communication Devices

Cheap Talk 4 Direct with Jacks (#1336)

Say It Rocking Plate (#1398)

Rocking Say It Play It (#1399)

Twin Talk (#1401)

Twin Talk & Play It (#1402)

Vocal Traffic Light (#1820)

Talking Bubble (#2220)

Talking Bubble Busy Box (#2230)

Cheap Talk 4 Inline With Jacks (#3054)

Cheap Talk 4 Inline With Jacks (#3055)

Cheap Talk 4 Inline Play It (#3056)

Cheap Talk 4 In-Line with Overlay Option (#6054)

Cheap Talk 4 In-Line with Jacks & Overlay Option
(#6055)

Cheap Talk 4 Square with Overlay Option (#6392)

Cheap Talk 4 Square with Jacks & Overlay Option (#6392)

Toys for Special Children, Inc.

385 Warburton Avenue
Hastings-on-Hudson, NY 10706
Telephone 914-478-0960 - Fax 914-478-7030

6-Level Communicator

Catalogue #2391

User's Guide

Toys For Special Children, Inc.

Hastings-on-Hudson, NY 10706