

Other Communication Devices

Communication Builder #7071

Communication Builder #7071-B (Black)

7 Level Blue Communication Builder #7075

7 Level Black Communication Builder #7075-B

1 Level-32 Message Communicator #3201

2 Level-32 Message Communicator #3202

3 Level-32 Message Communicator #3203

6 Level-32 Message Communicator #3206

Auditory Queuing Communicator 16 level #4399

2 Level Communicator with Overlay Option #2302

2 Level Communicator with 1/8" Jacks And Overlay Option #2303-8

6-Level Communicator with Overlay #2391

6-Level Communicator with 1/8" Jacks and Overlay Option #2392-8

Enabling Devices & Toys for Special Children

385 Warburton Avenue

Hastings-on-Hudson, NY 10706

Telephone 914-478-0960 / Fax 914-478-7030

www.enablingdevices.com

On-the-Go-Cheap Talk 8

Catalog # 1391-G

USER'S GUIDE

Enabling Devices & Toys For Special Children

Hastings-on-Hudson, NY 10706

www.enablingdevices.com

Introducing truly portable Communicators!

We have created a truly portable version of our best and most affordable communicators. The On-the-Go product line comes with a sturdy handle and 36" adjustable strap. Now you can take your communicator anywhere!

User-friendly teaching tool is great for providing communication and cognitive support, as well as enhancing the developing expressive and receptive language skills. Cheap Talk 8 allows you to record and rerecord eight messages with 37.5 seconds per message and 100% better sound quality than earlier versions. Each 2" square has an easy-to-remove plastic protective cover to hold words and/or pictures. Size: 12" x 7½" x 2 ¼ ". Requires 4 AA Batteries.

OPERATION:

1. Carefully turn over unit to reveal battery compartment. Observing polarity, install four AA batteries into the holder.
2. The microphone and red "**RECORD**" button are located on the back panel of the unit. To record, depress and hold the "**RECORD**" button, then depress one of the 8 colored squares and speak into the microphone. Release both buttons once recording is over. Playback your message by pressing and releasing the same square used to record. Each of the remaining buttons can be "programmed" in the same manner. (The unit will retain recorded information indefinitely, even if the battery is disconnected. Previously stored information will be erased only when new recording is performed.)
3. Playback volume can be adjusted using the control provided on the back panel of the unit, next to the "**RECORD**" button.
4. Turn the unit off by rotating the "**VOLUME**" control knob located on the back of the unit counterclockwise until you hear an audible "click".

IMPORTANT NOTES:

1. The memory of this unit is capable of holding up to 300 seconds of speech information total. This memory is divided in 8 equal segments of 37.5 seconds each.
2. **Cheap Talk 8** will shut off automatically after playback to preserve battery life. However, device still drains a very small current in this "sleep" mode. Therefore, if the unit is not going to be used for a prolonged amount of time, it is recommended to turn the power off. Doing this can extend overall battery life.
3. If the unit fails to operate, or operates incorrectly, the following steps should be performed:
 - Disconnect the power by removing one of the batteries for 20 seconds to reset the unit. If the unit still does not work correctly after battery is reconnected, replace the batteries observing proper polarity.
 - Make sure there are no obstacles between the colored square buttons and the housing unit.

If you have any problems or questions about this unit or any of our products, please call our Technical Assistance Department at:

(1-800-832-8697)

customer_support@enablingdevices.com

