

Troubleshooting:

Problem: The unit fails to operate, or operates incorrectly.

Action #1: Make sure there is no dirt or obstacles between the square plates and the housing of the unit.

Action #2: If the unit still does not work, disconnect the power, by removing one of the batteries for 20 seconds, to reset the unit. Reconnect the battery. If the unit still does not work correctly, replace all batteries.

Care of Unit:

The Cheap Talk 4 Inline with Overlay Option can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit as it will damage the electrical components.

Do not use abrasive cleaners as they will scratch the surface of the unit.

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697 or
customer_support@enablingdevices.com

Cheap Talk 4 Inline with Overlay Option

The Cheap Talk 4 Inline with Overlay Option is a self-contained communication device which allows the user to record and play back four different messages of up to five seconds each. This model has an insert overlay to facilitate the use of interchangeable picture cards, which can be slipped behind the switch plate face. The user simply touches one of the 2-inch square pictures to activate a pre-recorded message. The unit can be secured to any smooth surface using the attached suction cups.

Operation:

1. Carefully turn over unit and remove the battery case cover to reveal the battery compartment. Observing polarity, install four AA size batteries (Not included) into the holder. Alkaline batteries are recommended because they last longer. Do not use rechargeable batteries because they supply lower voltage and the unit may not perform as well.
2. The microphone and the red "RECORD" button are located on the back panel of the unit. To record, depress and hold the red "RECORD" button. At the same time, depress one of the four plates on the front of the unit and speak into the microphone.

3. Release both the plate and the recording button when recording is over. Play back your message by pressing and releasing the same plate used to record. Each of the remaining plates can be "programmed" in the same manner.

The unit will retain recorded information indefinitely, even if the battery is disconnected. Previously stored information will be erased only when a new recording is performed.

4. Draw or print corresponding icons or words on a sheet of paper measuring 10" x 2-3/4" so that they appear through the four cutouts on the housing when inserted.


5. Carefully insert the sheet between the two top cover plates. Slide the insert down until your icons or words appear through the four square windows.

6. Playback volume can be adjusted using the "ON/OFF/VOLUME" control knob located on the back panel of the unit.

Important Notes:

1. The maximum message length for any single plate is five seconds, up to a total of 20 seconds for all four plates. If desired, a longer message can be recorded using the first plate, but each segment over five seconds will use the memory of succeeding plates.

The plate sequence is as follows:

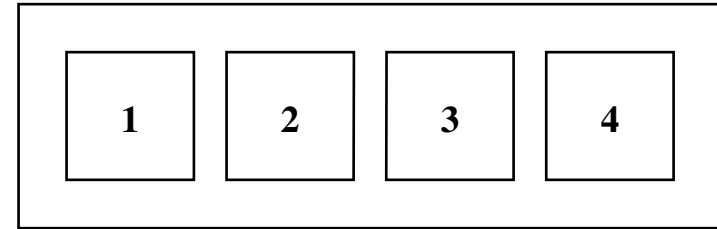

For example, a 15-second message would use the memory of plates 1-3, leaving only plate 4 free for recording.

Alternatively, a 15-second message could be stored using plate 2, in which case it will take the memory of plates 2-4, and another 5-second message could be stored at location 1. Bear in mind, however, that if message length exceeds available memory amount, it will wrap around over itself at its beginning.

2. Cheap Talk 4 Inline with Overlay Option will shut off automatically after playback, to preserve battery life. However, the device still drains a very small current in this "sleep" mode. Therefore, if the unit is not going to be used for a few days, it is recommended that the unit be turned off, to extend overall battery life. This can be done by turning the "ON/OFF/VOLUME" switch counterclockwise, until it "clicks" off.

More Communication Products

Rocking Recorder #1333

Cheap Talk 4 Direct & Jacks #1336

Cheap Talk 4 Direct #1392

Rocking Say It #1398

Rocking Say It Play #1399

Twin Talk Recorder with Divider #1444

Vocal Traffic Light #1820

Talking Bubble #2220

Talking Bubble Busy Box #2230

Finger Isolation Talking Bubble #2260

Cheap Talk 4 Direct Inline & Jacks #3055

Cheap Talk 4 Inline Say It Play It #3056

Cheap Talk 4 Direct Touch with Overlay Option #6392

Toys for Special Children, Inc.

385 Warburton Avenue

Hastings-on-Hudson, NY 10706

Telephone 914-478-0960 - Fax 914-478-7030

Cheap Talk 4 Inline with Overlay Option

Catalog #6054

User's Guide

Enabling Devices, Inc.

Hastings-on-Hudson, NY 10706

www.enablingdevices.com