

---

---

**Smurf #8044**

---

---

**USER'S GUIDE**

---

**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-TEC-TOYS or 1-800-832-8697  
customer\_support@enablingdevices.com

---

  
**Enabling Devices**

385 Warburton Avenue  
Hastings-on-Hudson, NY 10706  
Tel. 914.478.0960 / Fax 914.478.7030  
[www.enablingdevices.com](http://www.enablingdevices.com)

*They are back!*

Experience the magical world of Smurfs. Just add your capability switch and hear the wonderful phrases that it says to you. DVD and a music video included. Smurfs may vary. Size: 12"H x 5"W x 5"D. Requires 3 AG13/LR44 button cell batteries included.

**Operation:**

1. Smurf require 3 LR44 watch type Batteries to operate. Alkaline batteries must be used (e.g. Duracell or Energizer Brands). Never mix old and new batteries together or different brands or types together.
2. The battery compartment is located in the doll's back. Open the Velcro closure. Next using a Phillips head screwdriver remove the screw and install 3 LR44 batteries paying close attention to proper (+) and (-) battery polarity. Replace the battery compartment cover and secure it with the screw.
3. Connect Smurf to any external capability switch (not included) via the cord jack. Smurf is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure you have a tight connection. There should be no gaps.
4. Actuate your external capability switch and Smurf will come to life! It will speak different to you.
5. Smurf also come with a removable base stand. They connect to the base via the Velcro strips attached to their back and stand.

**Troubleshooting:**

**Problem:** Smurf does not function correctly.

**Action #1:** Replace the batteries paying close attention to proper (+) and (-) battery polarity.

**Action #2:** Make sure that the connection between the external capability switch and the jack is TIGHT. There should be NO gaps.

**Action #3:** Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

**Care of Unit:**

If you will not be using Smurf for an extended amount of time, we recommend removing the batteries.

**Do not submerge** Smurf in water, as it will damage the electrical components. Smurf is surface washable only.