
**Rocking Two Message Communicator
with Light- #1409**

USER'S GUIDE

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697
customer_support@enablingdevices.com


Enabling Devices

385 Warburton Avenue
Hastings-on-Hudson, NY 10706
Tel. 914.478.0960 / Fax 914.478.7030
www.enablingdevices.com

Visual cue when message is played!

Lights go on when message is played. Records two messages (10 seconds per message). Two 2" x 2" icons are attached to the switch plate, so you can put a picture or message. Size: 5"L x 6"W x 1 1/2"H. Requires 4 AA Batteries (Not included).

Operation:

1. Carefully turn over unit and remove the small screw from the battery compartment. Slide the battery compartment cover off. Observing polarity, install four AA size batteries (Not included) into the holder. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. Turn the unit on by turning the ON/OFF/VOLUME knob located on the back panel of the unit. To record, simply press and "hold" the red "RECORD" button, then hold down one side of the plate. While continuing to hold the plate down, speak your message. Release both REC button and plate when you finish recording. Record the other side of the plate in the same manner.
3. To play back your message, press lightly on the side of your choice. It is not necessary to continue holding the plate down when listening to the message. The unit will continue to playback the entire message, even if pressure on the plate is released. While your message plays the Red LED lights will illuminate.
4. The unit will retain recorded information indefinitely, even if the batteries are disconnected. Previously stored information will be erased only when a new recording is performed.
5. The two, icon holders provide ample space for words, symbols or pictures. These visuals should be cut or designed to fit in the square inset area of each plate, and then covered with the clear overlays provided. To do this, simply bend the clear plastic overlay and slip the sides into the slots.

6. Playback volume can be adjusted using the "ON/OFF/VOLUME" control knob located on the back panel of the unit.

Important Notes:

The memory of this unit is capable of holding up to a total of 20 seconds of speech information.

Troubleshooting:

Problem: The unit fails to operate, or operates incorrectly.

Action #1: Make sure there is no dirt or obstacles between the plate and the housing of the unit.

Action #2: Disconnect the power by removing one of the batteries for 20 seconds in order to reset the unit. Re-install the batteries. If the unit does not work correctly, replace all the batteries.

Care of the Unit:

The Rocking Two Message Communicator can be wiped clean with any household multi-purpose, non-abrasive cleaner and disinfectant.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.

Care of Unit:

Lingo Bingo can be wiped clean with any mild household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit