



Photos. 3 & 4

**enabling**  
devices

**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697

[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)

---

---

## Wobble Switch on Gooseneck #1063

---

---

### USER'S GUIDE

**enabling**  
devices

50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

We're pleased to offer you our Wobble Switch on a 19" gooseneck with an adjustable wheelchair clamp. *Weight: 2¼ lbs.*

### Operation:

1. Using the mounting clamp, attach the Wobble Switch to a Wheel Chair, Table or Desk. Tighten the clamp enough to hold the Wobble Switch securely, but do not over-Tighten the clamp. (See Photos).
2. Plug the Wobble Switch into your switch adapted toy/device. Make sure that the connection is plugged in all the way. There should be NO gaps. If you use an adapter for devices with ¼ inch jacks, make sure that it is a MONO adapter, not stereo.
3. Actuate the Wobble switch in order to turn on your toy/device. Your toy/device will only remain activated while you hold the Wobble switch in the actuated position. Once you release pressure on the Wobble switch, the toy/device will turn off.
4. **Clamping Options:** Our mounting clamp is designed for mounting to tubular (wheelchair, bedrail, walker, etc.) or flat surfaces such as a table top or desk. For flat surface mounting use the included wedge insert shoe stored in front of the locking knob on the clamp (see Photos.1 & 2) Loosen locking knob slightly to remove wedge, tighten knob once wedge has been removed.



Photo.1



Photo. 2

5. Using the mounting clamp, attach the Wobble Switch to a Wheel Chair, Table or Desk (See photos 3 & 4 on back page of guide). Tighten the clamp enough to hold the Ultimate Switch securely, but do not over-Tighten the clamp.

### Troubleshooting:

**Problem #1:** The toy/device is not activating.

**Action #1:** Make sure that the connection to the toy/device is plugged in all the way. There should be NO gaps. Make sure that you are using a MONO adapter, if applicable.

**Action #2:** Try using your toy/device with a different switch in order to rule out the toy/device as the source of the problem. You may need to change the batteries in the toy/device.

### Care of Unit:

The Wobble Switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

**Do not submerge** the unit, as it will damage the electrical components.

**Do not use abrasive cleaners**, as they will scratch the surface of the unit.