
Wobble Switch On Base #1070

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Our popular Wobble Switch mounted vertically on a base and secured with four rubber suction cups. Size: 7"L x 7"W x 7"H. Weight ½ lb.

Operation:

1. Plug the Wobble Switch into your switch adapted toy/device. Make sure that the connection is tight. There should be NO gaps. If you use an adapter for devices with ¼ inch jacks, make sure that it is a MONO adapter, not stereo.
2. Actuate the Wobble switch in order to turn on your toy/device. Your toy/device will only remain activated while you hold the Wobble switch in the actuated position. Once you release pressure on the Wobble switch, the toy/device will turn off.

Troubleshooting:

Problem #1: The toy/device is not activating.

Action #1: Make sure that the connection to the toy/device is plugged in all the way. There should be NO gaps. Make sure that you are using a MONO adapter, if applicable.

Action #2: Try using your toy/device with a different switch in order to rule out the toy/device as the source of the problem. You may need to change the batteries in the toy/device.

Care of Unit:

The Wobble Switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.