Butterfly #148

USER'S GUIDE



50 Broadway Hawthorne, NY 10532 Tel. 914.747.3070 / Fax 914.747.3480 Toll Free 800.832.8697 www.enablingdevices.com



For Technical Support:

Call our Technical Service Department Monday through Friday, 9 a.m. to 5 p.m. (EST) 1-800-832-8697

customer support@enablingdevices.com

Plays 18 songs!

Activate your capability switch and the multi-colored blinking lights dance around the Butterfly while music plays. Great for visual tracking. Size: 51/4"L x 41/2"W x 2"H. Requires Capability Switch and 2 AA Batteries. Weight: 1/2 lb.

Operation:

- 1. Gently turn the Butterfly over to reveal the battery compartment. Carefully remove the small screw from the battery compartment cover using a Phillips head screwdriver, and slide the battery compartment cover off. Observing polarity, install two AA size batteries into the holder. Alkaline batteries must be used only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Replace the battery compartment cover and replace the screw.
- 2. Plug any capability switch into the 1/8" cord jack located on the side of the Butterfly, make sure the switch is plugged in all the way. There should be no gaps between the connections, if there is a gap the Butterfly will stay ON and your switch will have no control over the Butterfly.
- 2. Each time you activate your switch you will be rewarded with a well know children's song accompanied by flashing colored lights in tune with the music. The song will play between 15 and twenty seconds depending on the song. If your switch is activated during the playing of a song, it will advance to the next song. You can also play with the Butterfly by pushing the Butterfly s Bowtie. Some of the songs you may here are BINGO, Mary Had a Little Lamb, The Farmer and the Dell, This Old Man etc.

Troubleshooting:

Problem: The unit fails to operate, or operates incorrectly.

Action #1: Disconnect the power by removing one of the batteries for 20 seconds in order to reset the unit. Re-install the batteries.

Action #2: Check your batteries – There is a 99% chance that either one is installed incorrectly and/or one is weak/dead.

Please Note: When you are finished using Butterfly remove your external switch as there is no ON/OFF power switch. This will help prevent false activations and conserve battery life. For prolonged storage we recommend removing the batteries to prevent battery drain and or battery leakage which can cause damage to Butterfly's electronics.

Care of the Unit:

Butterfly can be wiped clean with any household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.