

---

---

## Attendant Call Chime #1499

---

---

### USER'S GUIDE



**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697

[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

### Alerts Attendants Immediately!

You rang? Our easy-to-use switch adapted call chime lets you know the user needs assistance, even if you are as far away as 1,000 feet! The Attendant Call Chime comes with one switch adapted transmitter and two receivers that plug into any 120V standard wall outlet. Receivers have volume control and multiple chime or music options. Size: Receiver: 4½"L" x 2½"W x 1½"D; Transmitter: 2½"L"x 1 ¼"W x ½"D. Requires 1 CR2032 Battery. Weight: ¼ lb.

### Operation:

1. Follow the included OEM user's Guide for setup.
2. Connect any capability switch, via the 1/8" cord jack on the transmitter. Make sure your connection between the two is plugged in all the way, with no gaps. If it is necessary for you to use an adapter for switches with the larger 1/4" plug, make sure that you use a MONO adapter like or No. 1170, not a STEREO adapter.
3. Place one or both Chiming Receivers in the best location to be heard within the 1000 ft. range. When the need arise to get a caregiver's attention simply activate your external capability switch. This will cause the receivers to chime alerting the caregiver. The chime will sound for as long as your switch is activated. **Please note: the maximum range of this device under ideal conditions is 1000 feet.**
6. There are many other useful ways to use this handy device. **Please Note:** you cannot use two of these devices in the same room; doing so will cause activation of both Chiming receivers when one transmitter has been activated.

### Troubleshooting:

**Problem:** The Attendant Call Chime does not activate.

**Action #1:** Make sure that all battery are installed following proper (+) and (-) polarity. Batteries must be fresh, if weak or dead replace.

**Action #2:** Make sure that the distance from the receiver and transmitter is no more than 1000 feet away at any given time. This is the maximum range of use for this device.

### Care of Unit:

The Attendant Call Chime can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

**Do not submerge** the unit, as it will damage the contents and the electrical components.

**Do not use abrasive cleaners**, as they will scratch the surface of the unit.