
Compact Activity Center #2300

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-832-8697

customer_support@enablingdevices.com



50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com

Specially designed for children with limited mobility!

This small-scale activity center is designed to provide sensory stimulation and cognitive activity. Great for the user whose physical mobility may be limited due to positioning needs. Features a bead curtain, a dome with brightly-colored sequins which spin when the button is pressed, a textured plate which vibrates and lights up when pressed, and a pull ball that turns on a music box. Requires 2 C Batteries. Size: 23½"L x 12½"W x 10½"H. Weight: 3½ lbs.

Operation:

1. The battery case is located behind the mirror. To remove the mirror, turn the screw on the bottom face of the mirror counterclockwise with a screwdriver. Next remove the small Phillip's screw from the batter case cover and slide the cover off. Observing polarity, install two C batteries into the holder. Use alkaline batteries only because. Do not use rechargeable batteries or heavy duty type batteries because they supply lower voltage and the unit may not perform as well... Reattach mirror being careful not to overtighten screw.
2. Set the push button ON/OFF switch located on the side of the unit to ON. When finished using turn the switch off to prevent false activation and battery drain.

Troubleshooting:

Problem: The unit fails to operate, or operates incorrectly.

Action: Replace batteries.

Care of Unit:

The Compact Activity Center can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit as it will damage the electrical components.

Do not use abrasive cleaners as they will scratch the surface of the unit.