

---

## Penguin Roller Coaster (Slide Toy) #300

---

### USER'S GUIDE



**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697

[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

You don't have to travel to the South Pole to see these adorable little penguins climb to the top of their iceberg and "swoosh" down the slide to the bottom. Increases visual attention and tracking skills. Works with your capability switch. Size: 7¼"L x 2½"W x 8¾"H. Requires Capability Switch and 3 AA Batteries. Weight: 1¼ lbs.

#### **Operation:**

1. Penguin Roller Coaster requires 3 AA batteries (Not included). Use an alkaline battery only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly.
2. The battery compartment is located on the side of the unit. To remove the battery compartment cover, unscrew the small Phillips Head screw, then pull the cover open. Install a new battery, being careful to observe proper (+) & (-) battery polarity. Replace cover and screw.
3. Set the ON/OFF switch to OFF for external switch use. The toy will function normally without an external switch for continuous play. Connect the unit to any external capability switch via the 1/8" cord jack, located on the side of the unit. If you need to use a ¼" to 1/8" adapter for your switch, it must be a mono adapter, not stereo.
4. Make sure that the connections between the external capability switch and the adapter (if applicable) and the jack on the slide toy are tight. There should be no gap.
5. After assembling the track (follow in instructions for assembly located on the side of the box cover), load the supplied Penguins into the track at the bottom of staircase. Make sure the slide toy is on a level table. Begin play by activating and maintaining activation of your external capability switch, this will start the slide toy picking up the Penguins one by one. Once a Penguin reaches the top of the staircase, it will then begin rolling down the track.
6. Once you release your switch the toy will stop. If you would like the toy to run for a set amount of time, or by activation your switch once to start and then a second time to stop it use one of our

switch latch timer modules (e.g.No.605) sold separately. There are also many of our switches that have a built in Latch/Timer function. Any of the switches in our catalog or website with a -X after the number has this feature.

#### **Troubleshooting:**

**Problem:** The slide toy does not work.

**Action #1:** Check or replace the 3 AA batteries with fresh new batteries.

**Action #2:** Make sure the slide toy is on a flat level surface and that nothing is restricting any of the bears or track. Use only the Penguins included with the slide toy, other slide toy pieces will not work with this product.

**Action #3:** Try a different switch that you know works with other toys/devices to rule out the slide toy as the source of the problem.

**Problem:** The slide toy runs non stop when I plug my switch into it.

**Action# 1:** Make sure your switch is plugged into the cord jack all the way, there should be no gap. If there is a gap the slide toy will run non stop.

#### **Care of Unit:**

The slide toy can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

**Do not submerge** the unit, as it will damage the contents and the electrical components.

**Do not use abrasive cleaners,** as they will scratch the surface of the unit.