



enabling
devices

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com

Tactile Switch Set #3111

USER'S GUIDE

enabling
devices

50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Great texture!

This set contains two wonderfully textured switches – one with vibration, one without. The slightest touch on the upper rim or tactile dome will activate any switch-enabled device. A wonderful somatosensory experience. Size: 4½"Diameter x 2½"H. Switch with vibration requires 2 AAA Batteries. Weight: ½ lb. each. **Please**

Note: Textured Top color will vary.

Operation:

1. The Vibrating switch requires 2 AAA batteries to operate (Not included). Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. Turn the unit over to face the battery compartment cover. Turn the Lock/Unlock key to Unlock with a flathead screwdriver or a quarter. Pop the cover off. (See photos on back of guide). Observing proper (+) & (-) battery polarity, install 2 AAA size batteries into the holder. Replace the battery compartment cover. Set the On/Off push button switch located on the top of the switch to ON if vibration is desired when the switch is activated. This switch can also be used without the vibration by turning it OFF.
3. Plug either switch into the jack on your switch-adapted toy/device.
4. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
5. Make sure that you plug the switch plug all the way into your toy/device. There should be no gaps.
6. Simply press anywhere on the switch's textured top to activate your toy/device.

7. The toy/device will only remain activated while the Textured switch continues to be actuated. Once you release the switch, the toy/device will turn off. The user will also feel a gentle vibration from the switch as long as the switch is being activated (Vibration switch only).

Troubleshooting:

Problem: The Textured switch does not activate your toy/device.

Action #1: Make sure that the connection between the Textured switch and your toy/device is plugged in all the way. There should be **no** gaps. This is a common error and an easy fix.

Action #2: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Action #3: Try a different adapter (if applicable) to rule this out as the source of the problem.

Action #4: Make sure the batteries are fresh (Vibration switch); this switch's vibration will not work without batteries.

Care of Unit:

The Textured switches can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.