
**Fun Animal Switches- #3157M, (Ming), V
(Vines), S (Speckles), P (Slo Poke) & 3158
Set of 4 Switches**

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697
customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Easy activation!

Kids will love these fun, soft animal switches. The slightest press or squeeze on the tummies of these fun animals will activate any adapted device. Great for engaging children in active play. Choose from four different animals or buy all four and save. Surface wash only. Each weighs ¼ lb. Size: 3½"L x 3½"W x 7"H.

Operation:

1. We've included a small, light touch switch with adhesive backed Velcro to place on the pillow to create an easily accessible lap switch.
2. Plug the Animal Switch into the switch input jack on your switch-adapted device.
3. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter
4. Make sure that the Animal Switch is plugged in all the way. There should be no gaps.
5. The slightest press or squeeze on the tummies of these fun animals will activate any adapted device. Great for engaging children in active play. The device will only remain activated while the Animal Switch continues to be actuated. Once you release the Animal Switch, the device will turn off.

Troubleshooting:

Problem: The Animal Switch does not activate your toy/device.

Action #1 Make sure that the Animal Switch is plugged in all the way. There should be no gaps. This is a common error and an easy fix.

Action #2: If you are using an adapter, make sure that it is a MONO adapter, not a STEREO adapter.

Action #3: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Care of Unit:

The Animal Switch is surface cleanable only.

Do not submerge the unit in any liquids, as it will damage the electrical components.