



Fig.3

Clip Clamp #3706

USER'S GUIDE

enabling
devices

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com

enabling
devices

50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Clip clamps are oh so convenient and have so many uses! Great for positioning items including Eye Talks, photos or icons where they are easy to see, and easily mounts on wheelchair trays, tabletops or bedrails. Comes with a 24" flexible tubing, super clamp and hand clamp. Size: 27"H. Weight: 2¼ lbs. NOTE: Eye-talk not included.

1. **Clamping Options:** Our mounting clamp is designed for mounting to tubular (wheelchair, bedrail, walker, etc.) or flat surfaces such as a table top or desk. (See photos.1, 2 & 3) Using the mounting clamp, attach the Clip Clamp to a Wheel Chair, Table or Desk. Tighten the clamp enough to hold the Clip Clamp securely, but Do Not Over-Tighten the clamp as damage may occur.



Fig.1



Fig. 2

Please Note: The maximum weight the arm is designed to hold is between 1 to 2 lbs. Do not exceed this weight limit or damage to the arm and your item can occur.

Care of Unit:

Clip Clamp can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.