

---

## Tail Light Switch Say-It-Play-it

---


### Versatile!

Touch anywhere on this switch - even the rim - ever so lightly and hear your pre-recorded message accompanied by vibration and lights. Plug in your device and it will start when the switch is touched. Total record time is 20 seconds. Can be used on a tabletop or mounted on a door or wall. Wall hanger (#466) sold separately. Size: 7¼"W x 2½"H. Requires 4 AA Batteries. Weight: 1 lb.

### Operation:

1. Carefully turn over unit and remove the battery case covers to reveal battery compartments. Observing polarity, install four AA size batteries (Not included) into the holders. Alkaline batteries are recommended because they last longer. Do not use rechargeable batteries because they supply lower voltage and the unit may not perform as well.
2. The microphone and the red "RECORD" buttons are located on the back panel of the unit. To record, first depress and hold the red "RECORD" button marked No.1 in the photo. Then while holding the No.1"RECORD" button, depress the No.2 padded RECORD buttons (see photo) and speak into the microphone. Release both the padded button and the recording button when the recording is over. Place the Tail Light on a flat surface, or use the optional wall mount. Press anywhere on the Tail Light and your message will play, along with lights and vibration.
3. The unit will retain recorded information indefinitely, even if the battery is disconnected. Previously stored information will be erased only when a new recording is performed.
4. Playback volume can be adjusted using the "ON/OFF/VOLUME" control knob located on the back panel of the unit.
5. To use as a switch, plug one end of the supplied double ended 1/8" to 1/8" cord into the jack located on the side of

the Tail Light. Next plug the other end into your switch adapted toy/device. Press anywhere on the Tail light, and your toy/device will activate along with your pre-recorded message, lights and vibration. Once you release the Tail Light your toy/device will shut off.

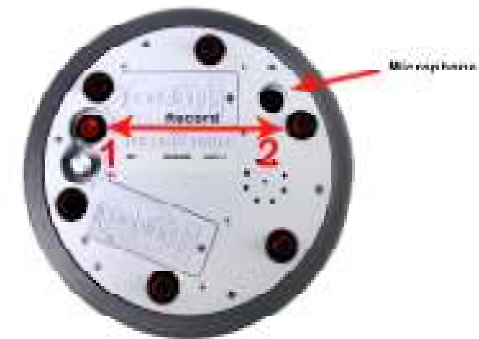

### Important Notes:

1. The memory of this unit is capable of holding up to a total of 20 seconds of speech information. If message length exceeds available memory amount (e.g., 20 seconds), it will wrap around over itself at its beginning.
2. The Tail Light Switch will shut off automatically after playback, to preserve battery life. However, the device still drains a very small current in this "sleep" mode. Therefore, if the unit is not going to be used for a few days, it is recommended that the unit be turned off, to extend overall battery life. This can be done by turning the "ON/OFF/VOLUME" switch counterclockwise, until it "clicks" off.

Revision 6-24-2022

## Troubleshooting:

**Problem:** The unit fails to operate, or operates incorrectly.

**Action #1:** Make sure that you are using the Tail Light on a flat surface.

**Action #2:** Disconnect the power, by removing one of the batteries from each compartment for 20 seconds, to reset the unit. Reconnect the battery. If the unit still does not work correctly, replace all batteries.

**Action # 3:** Make sure all connections between the Tail Light, cord and toy/device are plugged in all the way. There should be no gaps.

## Care of Unit:

The Tail Light Say-It-Play-It can be wiped clean with any household multi-purpose cleaner and disinfectant.

**Do not submerge** the unit as it will damage the electrical components.

**Do not use abrasive cleaners** as they will scratch the surface of the unit.



### For Technical Support:

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-TEC-TOYS or 1-800-832-8697  
[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)

---

# Tail Light Switch Say-It-Play-It #464

---

## User's Guide



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)