

Perfect for the non-speaker!

Our Textured Sequencers are perfect for the non-speaking adult or child. It allows the user to express a sequential series of your prerecorded messages (300 second total). Roll or press the sequencer once and your first message is spoken, roll or press again and the second message is heard, etc. The sequencer allows people with even the most limited communication skills to carry on a simple conversation. Ideal for a number of different purposes including sequencing, testing, directions, multiplication, memorization, medical instructions, storytelling and sing-alongs. Now available in two different models. Requires 4 AA Batteries. Size: 11"L x 4½"W x 2"H. Weight: 1½ lbs.

OPERATION:

1. Carefully turn over unit to reveal the battery compartments. Install 4 AA batteries (Not included), observing proper polarity (+) and (-), in battery holders.
2. All controls are located on the back panel of the unit. The **Microphone** is located on the left side of the unit. To record, flip the **RECORD/PLAY** switch to the **RECORD** position. Depress the **REC-MESS** button and speak 6 to 8 inches from the microphone. **Please Note:** do not move the Textured Roller while recording, doing so will prevent you from making a recording. When you are finished recording the first message, release the **REC-MESS** button. To record all subsequent messages, simply depress the **REC-MESS** button, and speak into the microphone, releasing the **REC-MESS** button after each message.
3. Messages can vary in length and the total record time is 300 seconds.
4. To play the messages, flip the **RECORD/PLAY** switch to the **PLAY** position. Rolling the Textured Roller in either direction will play the first message, the second activation will play the second message, and so on. To interrupt the messages and start at the beginning of the sequence, press the **RESET** button on the back of the unit. **Please Note:** You can check your recordings when finished, by simply flipping the **RECORD/PLAY** switch to play, and then press the **REC-MESS** button. Each time you press the **REC-MESS** button your messages will in the sequence they were recorded.
5. The unit will retain recorded information indefinitely, even if the battery is disconnected. Previously stored information will be erased only when new recording is performed.
6. **Please Note:** When re-recording messages, all messages on the unit will be erased.
7. To turn the unit off, turn the volume control knob counter-clockwise until you hear an audible "click".

IMPORTANT NOTES:

The memory of this unit is capable of holding up to 300 seconds of speech information total

Troubleshooting:

If the unit fails to operate, or performs incorrectly, the following steps should be performed:

- Disconnect the batteries for 30 seconds to reset the unit. If the unit still does not work after battery is reconnected, replace the batteries with new ones.
- Make sure there are no obstacles between the Textured Roller and the housing of the unit.

If you have any problems or questions about this unit or any of our products, please call our Technical Assistance Department at:

1-800-TEC-TOYS
(1-800-832-8697)

**Textured Roller Step Talking
Sequencer #653**

User's Guide


Enabling Devices

50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com