

Strike up the band!

Attach a musical instrument on to our striking musical stand. Activate your switch and it beats to any preset rhythm. Now everyone can be included in music time. Great for increasing eye-hand coordination. Speed is adjustable. Set includes cluster bells, castanets and a jingle stick. Size: 9"L x 6½"W x 4⅝"H. Requires Capability Switch and 8 AA Batteries. Weight: 1¾ lbs

Operation:

1. The Music Machine requires 8 AA batteries (Not included). Alkaline batteries are recommended because they last longer. Do not use rechargeable batteries because they supply lower voltage and the unit may not perform as well.
2. The battery compartments are located under the unit. Carefully turn the unit over. Remove the battery covers screws by turning them counterclockwise with a Phillips head screwdriver. Install batteries, being careful to observe proper battery polarity.
3. Replace the battery case covers and secure them with screws. Do not over-tighten.
4. Plug in your capability switch into the 1/8" jack. The striking arm will only move during switch activation. The speed or tempo, at which the striking arm moves, can be adjusted. Turning the black adjusting knob clockwise increases the speed, counterclockwise decreases speed.
5. Make sure that you have TIGHT connections between your switch and The Music Machine. There should be no gaps.
6. To attach your instrument, loosen the black knob of clamp that is connected to the black gooseneck. Do this only enough to accommodate your instrument. Tighten the knob just enough to hold the instrument securely. **Please Note: overtightening of the knob will cause damage to this unit.** There is a second knob on this clamp, which is used to adjust the angle of your instrument. Clockwise tightens the silver knob and counterclockwise loosens this knob.

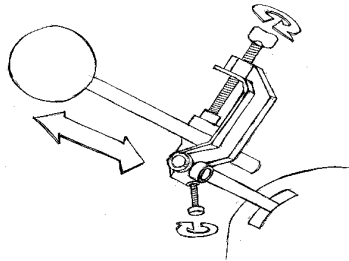


Fig.1

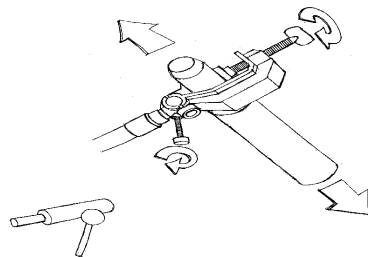


Fig. 2

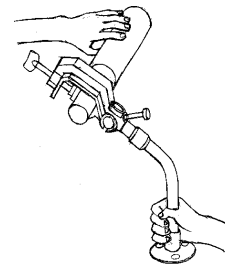


Fig. 3

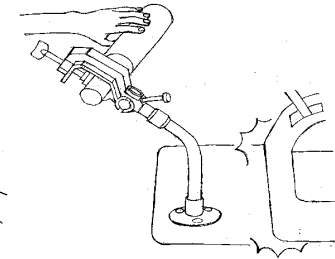


Fig. 4

7. At times it may be necessary to adjust the instrument gooseneck-mounting arm to accommodate your instrument. With one hand hold the base of the gooseneck, with your other hand bend the gooseneck to adjust the instruments placement. Please see Figs 1 thru 4.

Please Note: The speed adjustment is not linear on this product. You will only have about a 1/8th of an inch from the maximum speed setting to make any speed adjustments.

Troubleshooting:

Problem: There is no movement of the music arm, or the unit does not activate properly.

Action#1: Rotate (spin) the batteries within the battery compartment to ensure a good connection.

Action #2: Replace batteries paying close attention to proper battery polarity.

Action#3: Check to make sure you have tight connections between plug and jack. There should be NO gaps.

Action#4: Try The Music Machine with another external capability switch to rule it out as the source of the problem.

Care of Unit:

The Music Machine can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as this could damage the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

The Music Machine #703

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com