
**Oval Textured Switches
Small #711-S & Medium #711-M**

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-832-8697

customer_support@enablingdevices.com



50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com

Compelling oval textured switches invite exploration!

These brightly-colored oval switches are some of our most popular items. Appealing to all ages, they come in two sizes: Small (4½") Weight: ¼ lb., Medium (9") Weight: ½ lb.

Operation:

1. Plug the Oval Textured Switch into the jack on your switch-adapted toy/device.
2. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
3. Make sure that the connection between the Oval Textured Switch and your toy/device is tight. There should be no gaps.
4. Simply press anywhere on the textured yellow plate, to activate your toy/device.
5. The toy/device will only remain activated while the yellow plate is depressed. Once you release the yellow plate, the toy/device will turn off.

Troubleshooting:

Problem: The Oval Textured Switch does not activate your toy/device.

Action #1: Make sure that the connection between the Oval Textured Switch and your toy/device is tight. There should be *no* gaps. This is a common error and an easy fix.

Action #2: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Action #3: Try a different adapter (if applicable) to rule this out as the source of the problem.

Care of Unit:

The Oval Textured Switch can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.