

---

# Mini Saucer Switch #718-S

---

## USER'S GUIDE



**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697

[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

---

## Mini Saucer Switch

---

Ideal for those with limited motor abilities!  
These handy switches are specially designed for individuals who are unable to control or sustain the hand or wrist movements necessary to operate traditional plate switches. The Mini-Saucer and Saucer Switch are activated with just a light touch. The Adjustable Pressure Saucer Switch allows the user to adjust the pressure from approximately ½ to 32 ounces and has a small LED lamp that glows when the switch is activated. Size: 3¼"L x 3¼"W x 1"H. Weight: ¼ lb.

### Operation:

1. The Mini Saucer Switch does *not* require any batteries to operate.
2. Plug the Mini Saucer Switch into the jack on your switch-adapted toy/device.
3. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
4. Make sure that the connection between the Mini Saucer Switch and your toy/device is tight. There should be no gaps.
5. Simply press anywhere on the saucer switch to activate your toy/device.
6. The toy/device will only remain activated while the Mini Saucer Switch continues to be actuated. Once you release the switch, the toy/device will turn off.

### Troubleshooting:

**Problem:** The Mini Saucer Switch does not activate your toy/device.

**Action #1:** Make sure that the Mini Saucer Switch is on a flat surface (not tilted or vertical). This provides for optimal functioning.

**Action #2:** Make sure that the connection between the Mini Saucer Switch and your toy/device is tight. There should be *no* gaps. This is a common error and an easy fix.

**Action #3:** Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

### Care of Unit:

The Mini Saucer Switch can be wiped clean with any household multi-purpose cleaner and disinfectant.

**Do not submerge** the unit, as it will damage the contents and the electrical components.

**Do not use abrasive cleaners**, as they will scratch the surface of the unit.