
On/Off Jumbo Switch #7805

USER'S GUIDE

For Technical Support:
Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697
customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Easy to use!

This version of our popular Jumbo Switch works as an On/Off switch. Just press the switch to turn on a device and press it again to stop. Great for users who cannot sustain their hand on a switch. Size: 5 1/8" D x 1/2 "H

Operation:

1. The Jumbo Switch does ***not*** require any batteries to operate.
2. Plug the Jumbo Switch into the jack on your switch-adapted toy/device.
3. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
4. Make sure that the connection between the Jumbo Switch and your toy/device is tight. There should be no gaps.
5. Simply press anywhere on the Jumbo Switch's colored top to activate your toy/device.
6. The toy/device will remain activated. Press on the Jumbo switch again, and the toy/device will turn off.

Troubleshooting:

Problem: The Jumbo Switch does not activate your toy/device.

Action #1: Make sure that the connection between the Jumbo Switch and your toy/device is tight. There should be ***no*** gaps. This is a common error and an easy fix.

Action #2: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Action #3: Try a different adapter (if applicable) to rule this out as the source of the problem.

Care of Unit:

The Jumbo Switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.